
**activate
your product
warranty**

scan the QR code and
register your product
within 7 days to activate
your warranty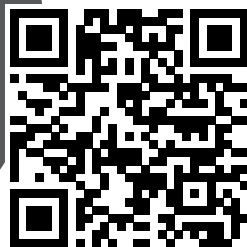

Registering also gives you access
to tips, exciting offers and more!

compact inversion back stretcher

INSTRUCTION MANUAL AND WARRANTY INFORMATION

BM-IS500J | L-04981, Rev. 1

IMPORTANT SAFEGUARDS

READ ALL INSTRUCTIONS BEFORE USING.


- NOT for use by children, and do not leave unattended children alone with this Back Stretcher.
- Use only for its intended use as described in this manual.
- Remove jewelry before using.
- Tie up long hair before using.
- The back stretcher should only be used by one person at a time.

SAVE THESE INSTRUCTIONS

CAUTION – PLEASE READ ALL INSTRUCTIONS CAREFULLY BEFORE USING.

- DO NOT use this product if you are pregnant
- Consult your doctor prior to using this product if:
 - You have a pacemaker
 - You have any concerns regarding your health
- Stop using the Back Stretcher immediately if you experience pain, dizziness, or nausea.
- DO NOT use the Back Stretcher for more than 5 minutes at a time.
- DO NOT use this product as a substitute for medical attention.
- This product should NEVER be used by any individual who has sensory deficiencies.

COMPONENTS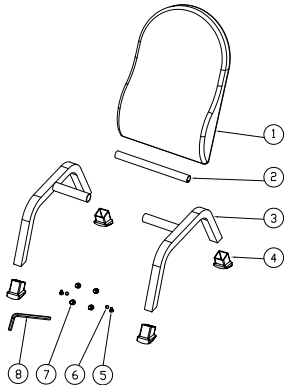


1. Backrest – 1
2. Rotating Shaft – 1
3. Support Frame Tubes – 2
4. Support Frame Rubber Foot Covers – 4 (pre-assembled)
5. 12mm Screws – 2
6. Washers – 2 (pre-assembled)
7. 16mm Connecting Bolts – 4 (pre-assembled)
8. Allen Wrench - 1


ASSEMBLY INSTRUCTIONS

For your convenience, the compact inversion stretcher has been partially assembled.


1. Insert the Rotating Shaft (Part 2), into the Backrest (Part 1).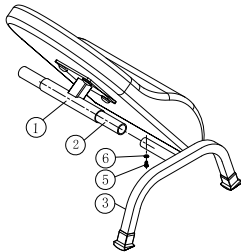

2. Connect a Support Frame Tube (Part 3) to each end of the Rotating Shaft. Insert the 12mm screws and use the Allen Wrench to tighten.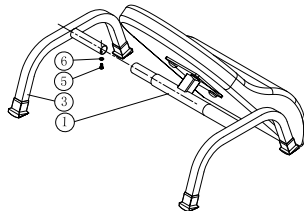


HOW TO USE

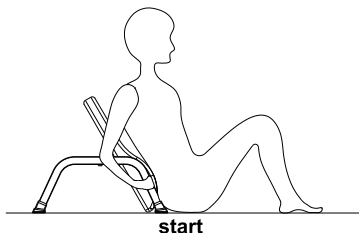
Position 1

Place the compact inversion back stretcher on a level floor. Sit down in front of the stretcher with your knees bent. Then use your hands to grip the support frames.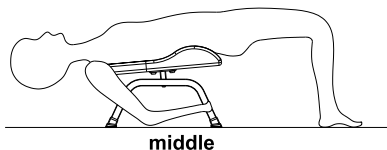

Position 2

Lean your body weight into the backrest and allow your torso to be lifted as shown in the drawing.

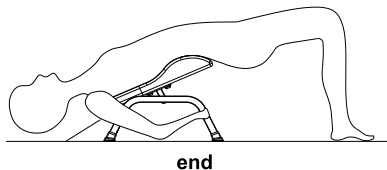Position 3

Continue leaning back until you are partially inverted. Hold this position for 1 - 5 minutes to enjoy a tension releasing stretch. Then lower your hips to return to position 1.

We recommend holding the stretch for 1 minute to begin with and then increasing the time, not to exceed 5 minutes, as you feel more comfortable. **Always listen to your own body and do what feels most comfortable to you!**

MAINTENANCE

TO STORE

Store in a cool, dry place.

TO CLEAN

- Use a damp towel to wipe clean. NEVER use abrasive cleaners, brushes, or strong chemicals that may or may not be damaging to the product.
- DO NOT attempt to repair the compact inversion back stretcher. There are no user-serviceable parts. For service, contact Consumer Relations at the telephone number listed in the Warranty section

2-YEAR LIMITED WARRANTY

Homedics sells its products with the intent that they are free of defects in manufacture and workmanship for a period of 2 years from the date of original purchase, except as noted below. Homedics warrants that its products will be free of defects in material and workmanship under normal use and service. This warranty extends only to consumers and does not extend to retailers.

To obtain warranty service on your Homedics product, contact a Consumer Relations representative for assistance. Please make sure to have the model number of the product available.

Homedics does not authorize anyone, including but not limited to retailers, the subsequent consumer purchaser of the product from a retailer or remote purchasers, to obligate Homedics in any way beyond the terms set forth herein. This warranty does not cover damage caused by misuse or abuse; accident; the attachment of any unauthorized accessory; alteration to the product; improper installation; unauthorized repairs or modifications; improper use of electrical/power supply; loss of power; dropped product; malfunction or damage of an operating part from failure to provide manufacturer's recommended maintenance; transportation damage; theft; neglect; vandalism; or environmental conditions; loss of use during the period the product is at a repair facility or otherwise awaiting parts or repair; or any other conditions whatsoever that are beyond the control of Homedics.

This warranty is effective only if the product is purchased and operated in the country in which the product is purchased. A product that requires modifications or adoption to enable it to operate in any other country than the country for which it was designed, manufactured, approved and/or authorized, or repair of products damaged by these modifications is not covered under this warranty.

THE WARRANTY PROVIDED HEREIN SHALL BE THE SOLE AND EXCLUSIVE WARRANTY. THERE SHALL BE NO OTHER WARRANTIES EXPRESS OR IMPLIED, INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS OR ANY OTHER OBLIGATION ON THE PART OF THE COMPANY WITH RESPECT TO PRODUCTS COVERED BY THIS WARRANTY. HOMEDICS SHALL HAVE NO LIABILITY FOR ANY INCIDENTAL, CONSEQUENTIAL, OR SPECIAL DAMAGES. IN NO EVENT SHALL THIS WARRANTY REQUIRE MORE THAN THE REPAIR OR REPLACEMENT OF ANY PART OR PARTS THAT ARE FOUND TO BE DEFECTIVE WITHIN THE EFFECTIVE PERIOD OF THE WARRANTY. NO REFUNDS WILL BE GIVEN. IF REPLACEMENT PARTS FOR DEFECTIVE MATERIALS ARE NOT AVAILABLE, HOMEDICS RESERVES THE RIGHT TO MAKE PRODUCT SUBSTITUTIONS IN LIEU OF REPAIR OR REPLACEMENT.

This warranty does not extend to the purchase of opened, used, repaired, repackaged, and/or resealed products, including but not limited to sale of such products on internet auction sites and/or sales of such products by surplus or bulk resellers. Any and all warranties or guarantees shall immediately cease and terminate as to any products or parts thereof that are repaired, replaced, altered, or modified, without the prior express and written consent of Homedics.

This warranty provides you with specific legal rights. You may have additional rights that may vary from state to state and country to country. Because of individual state and country regulations, some of the above limitations and exclusions may not apply to you.

For more information regarding our product line in the USA, please visit www.homedics.com. For Canada, please visit www.homedics.ca.

FOR SERVICE IN THE USA

cservice@homedics.com

8:30am-7:00pm EST Monday-Friday

1-800-466-3342

FOR SERVICE IN CANADA

cservice@homedics.ca

8:30am-5:00pm EST Monday-Friday

1-888-225-7378

©2024 Homedics, LLC. All rights reserved.

Homedics is a registered trademark of Homedics, LLC.

Distributed by Homedics, LLC, 3000 N Pontiac Trail, Commerce Township, MI 48390

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©2024 Homedics, LLC. Todos los derechos reservados.
Homedics es una marca registrada de Homedics, LLC.
Distribuido por Homedics, LLC, 3000 N Pontiac Trail, Commerce Township, MI 48390