
**activate
your product
warranty**

scan the QR code and
register your product
within 7 days to activate
your warranty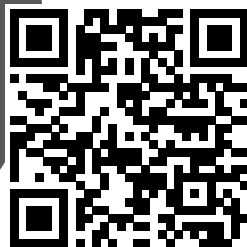

Registering also gives you access
to tips, exciting offers and more!

therapy percussion massager
INSTRUCTION MANUAL AND WARRANTY INFORMATION

IMPORTANT SAFETY INSTRUCTIONS

WHEN USING ELECTRICAL PRODUCTS, ESPECIALLY WHEN CHILDREN ARE PRESENT, BASIC SAFETY PRECAUTIONS SHOULD ALWAYS BE FOLLOWED, INCLUDING THE FOLLOWING:

READ ALL INSTRUCTIONS BEFORE USING.

DANGER – TO REDUCE THE RISK OF ELECTRIC SHOCK:

- Always unplug the appliance from the electrical outlet immediately after using and before cleaning.
- DO NOT reach for an appliance that has fallen into water. Unplug immediately.
- DO NOT use while bathing or in a shower.
- DO NOT place or store appliance where it can fall or be pulled into a tub or sink.
- DO NOT place in or drop into water or other liquid.

WARNING – TO REDUCE THE RISK OF BURNS, ELECTRIC SHOCK, FIRE, OR INJURY TO PERSONS:

- An appliance should NEVER be left unattended when plugged in. Unplug from outlet when not in use, and before putting on or taking off parts or attachments.
- Close supervision is necessary when this appliance is used by, on, or near children, invalids, or disabled persons.
- Use this appliance only for its intended use as described in this manual. DO NOT use attachments not recommended by Homedics; specifically, any attachments not provided with the unit.
- NEVER operate this appliance if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water. Return the appliance to a Homedics Service Center for examination and repair.
- Keep the cord away from heated surfaces.
- NEVER drop or insert any object into any opening.
- DO NOT operate where aerosol (spray) products are being used or where oxygen is being administered.
- DO NOT operate under a blanket or pillow. Excessive heating can occur and cause fire, electric shock, or injury to persons.
- DO NOT carry this appliance by the supply cord or use cord as a handle.
- To disconnect, turn all controls to the off position, then remove plug from outlet.
- DO NOT use outdoors.
- NEVER operate the appliance with the air openings blocked. Keep the air openings free of lint, hair, and the like.

- NEVER operate on a soft surface such as a bed or couch where the air openings may be blocked.
- Keep long hair away from massager while in use.
- DO NOT use massager in close proximity to loose clothing or jewelry.
- Recharge only with the charger provided with the unit. A charger that is suitable for one type of battery pack may create a risk of fire when used with another battery pack.
- DO NOT expose a battery pack or appliance to fire or excessive temperatures. Exposure to fire or temperature above 265°F may cause explosion.
- Follow all charging instructions and do not charge the battery pack or appliance outside of the temperature range specified in the instructions. Charging improperly or at temperatures outside of the specified range may damage the battery and increase the risk of fire. Product operating and charging range: 0°C – 40°C.
- This product has an internal, nonreplaceable lithium-ion battery. This battery is not user-serviceable. Please dispose in accordance with local, state, province, and country regulations.

SAVE THESE INSTRUCTIONS

CAUTION – PLEASE READ ALL INSTRUCTIONS CAREFULLY BEFORE OPERATING.

- This product is not intended for medical use.
- Consult your doctor prior to using this product, if:
 - You are pregnant
 - You have a pacemaker
 - You have any concerns regarding your health
- NOT recommended for use by people with diabetes.
- NEVER leave the appliance unattended, especially if children are present.
- NEVER cover the appliance when it is in operation.
- DO NOT use this product for more than 15 minutes at a time.
- Extensive use could lead to the product's excessive heating and shorter life. Should this occur, discontinue use and allow the unit to cool before operating.
- NEVER use this product directly on swollen or inflamed areas or skin eruptions.
- DO NOT use this product as a substitute for medical attention.
- DO NOT use this product before bed. The massage has a stimulating effect and can delay sleep.
- NEVER use this product while in bed.
- This product should NEVER be used by any individual suffering from any physical

ailment that would limit the user's capacity to operate the controls or who has sensory deficiencies.

- This unit should not be used by children or invalids without adult supervision.
- NEVER use this product in automobiles.
- This appliance is intended for household use only.
- NEVER use this appliance on or near eyes or other highly sensitive areas.
- All servicing of this unit must be performed by authorized Homedics service personnel only.

INSTRUCTIONS FOR USE

1. Your massager, when opened, may require charging.
To charge your massager, plug one side of the USB-C cord into the unit and plug the other end into an adapter. Next, plug the adapter into a 120-volt outlet. Your massager will be fully charged after a maximum of 2 hours charge time, and the charge will last up to 2 hours.
NOTE: This device only comes with a USB-C cord, not an adapter. A USB-C phone charger will also charge the unit. The unit will not operate when it is charging. Use a 5V2A (or 5V/1A, slower) charger, but can not be higher than 5V.
2. To check on remaining power in massager, press and hold power button for 3 seconds until LED display turns on. LED display will show amount of power as indicated in Fig. 1
3. To turn the massager on, press and hold the power button for 3 seconds.
4. Press the power button once to begin the massage and continue to press (up to 20 times) to increase intensity. Once through intensity cycle, press again to stop the percussion motion and again to resume on low intensity.
5. Apply the massage head to the muscle you would like to massage.
6. When the massage is complete, turn the unit off by pressing and holding the power button for 2 seconds. The unit is off when all LED lights turn off.

NOTE: This unit is equipped with a 15-minute auto-off feature for your safety.

Fig. 1

NOTE: All attachments are screw-on. Please do not pull or push the attachments into the massage gun.

TRIGGER POINT ATTACHMENT

Pinpoint problem areas and release trigger points. Also suitable for small pressure points like hands and feet.

BALL ATTACHMENT

Perfect attachment for overall use. This attachment can be used on small and large muscle groups.

U-SHAPED ATTACHMENT

Attachment is great for pinpointing muscles along the spine as well as around joints.

FLAT ATTACHMENT

Small surface area attachment allows for targeted massage. Best used if focused on areas such as the IT band, hamstrings, and chest.

MAINTENANCE

TO CLEAN:

- Be sure to unplug the unit and allow it to cool before cleaning. Use a soft, slightly damp cloth to wipe.
- NEVER allow water or any liquids to come into contact with the handle.
- This product has an internal, nonreplaceable lithium-ion battery. This battery is not user-serviceable. Please dispose in accordance with local, state, province, and country regulations.

TO STORE:

- Store this unit in its box or a cool, dry place.
- DO NOT hang the unit by the power cord.
- The product should be stored at 25°C (77 °F).

LOCATION OF PARTS AND CONTROLS

SUPPLIER'S DECLARATION OF CONFORMITY

Product Description: THERAPY PERCUSSION MASSAGER

Model Number: HHG-685

Trade Name: Homedics

FCC Compliance Statement

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) This device must accept any interference received, including interference that may cause undesired operation.

U.S. Contact Information

Company: Homedics, LLC.

Address: 3000 N Pontiac Trail, Commerce Township, MI 48390

8:30am-7:00pm EST Monday-Friday 1-800-466-3342

Homedics is not responsible for any radio or TV interference caused by unauthorized modifications to this equipment. Such modifications could void the user authority to operate the equipment.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment to an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

CAN ICES-003(B)/NMB-003(B)

1-YEAR LIMITED WARRANTY

Homedics sells its products with the intent that they are free of defects in manufacture and workmanship for a period of 1 year from the date of original purchase, except as noted below. Homedics warrants that its products will be free of defects in material and workmanship under normal use and service. This warranty extends only to consumers and does not extend to Retailers.

To obtain warranty service on your Homedics product, contact a Consumer Relations representative for assistance. Please make sure to have the model number of the product available.

Homedics does not authorize anyone, including, but not limited to, Retailers, the subsequent consumer purchaser of the product from a Retailer or remote purchasers, to obligate Homedics in any way beyond the terms set forth herein. This warranty does not cover damage caused by misuse or abuse; accident; the attachment of any unauthorized accessory; alteration to the product; improper installation; unauthorized repairs or modifications; improper use of electrical/power supply; loss of power; dropped product; malfunction or damage of an operating part from failure to provide manufacturer's recommended maintenance; transportation damage; theft; neglect; vandalism; or environmental conditions; loss of use during the period the product is at a repair facility or otherwise awaiting parts or repair; or any other conditions whatsoever that are beyond the control of Homedics.

This warranty is effective only if the product is purchased and operated in the country in which the product is purchased. A product that requires modifications or adoption to enable it to operate in any other country than the country for which it was designed, manufactured, approved and/or authorized, or repair of products damaged by these modifications is not covered under this warranty.

THE WARRANTY PROVIDED HEREIN SHALL BE THE SOLE AND EXCLUSIVE WARRANTY. THERE SHALL BE NO OTHER WARRANTIES EXPRESS OR IMPLIED INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS OR ANY OTHER OBLIGATION ON THE PART OF THE COMPANY WITH RESPECT TO PRODUCTS COVERED BY THIS WARRANTY. HOMEDICS SHALL HAVE NO LIABILITY FOR ANY INCIDENTAL, CONSEQUENTIAL, OR SPECIAL DAMAGES. IN NO EVENT SHALL THIS WARRANTY REQUIRE MORE THAN THE REPAIR OR REPLACEMENT OF ANY PART OR PARTS WHICH ARE FOUND TO BE DEFECTIVE WITHIN THE EFFECTIVE PERIOD OF THE WARRANTY. NO REFUNDS WILL BE GIVEN. IF REPLACEMENT PARTS FOR DEFECTIVE MATERIALS ARE NOT AVAILABLE, HOMEDICS RESERVES THE RIGHT TO MAKE PRODUCT SUBSTITUTIONS IN LIEU OF REPAIR OR REPLACEMENT.

This warranty does not extend to the purchase of opened, used, repaired, repackaged and/or resealed products, including but not limited to sale of such products on internet auction sites and/or sales of such products by surplus or bulk resellers. Any and all warranties or guarantees shall immediately cease and terminate as to any products or parts thereof which are repaired, replaced, altered, or modified, without the prior express and written consent of Homedics.

This warranty provides you with specific legal rights. You may have additional rights which may vary from state to state and country to country. Because of individual state and country regulations, some of the above limitations and exclusions may not apply to you.

For more information regarding our product line in the USA, please visit www.Homedics.com. For Canada, please visit www.Homedics.ca.

FOR SERVICE IN THE USA

cservice@Homedics.com

8:30am-7:00pm EST Monday-Friday

1-800-466-3342

FOR SERVICE IN CANADA

cservice@Homedics.ca

8:30am-5:00pm EST Monday-Friday

1-888-225-7378

©2024 Homedics, LLC. All rights reserved.

Homedics is a registered trademark of Homedics, LLC.

Distributed by Homedics, LLC, 3000 N Pontiac Trail, Commerce Township, MI 48390

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Compañía: Homedics, LLC.