Homedics

700 series 5-day trend
upper arm blood pressure monitor

INSTRUCTION MANUAL

BPA-100TRBT
BPA-100TRBTA
Doc #L-03087, Rev.4

5 year
Limited warranty

IMPORTANT PRODUCT NOTICES AND SAFETY INSTRUCTIONS

When using your blood pressure monitor, basic precautions should always be followed. Please read and follow all instructions and warnings before using this product. Save these instructions for future reference. This device is intended for over-the-counter home use in adults 18 years and older with an arm circumference ranging from 9 inches to 17 inches (approx. 23cm to 43cm).

The device can accurately measure blood pressure in pregnant patients including those with known or suspected in preeclampsia condition.

This device detects the appearance of irregular heartbeats during measurement; an indication symbol will appear with measuring reading. The BP Category Indicator will show the information with the readings on the screen for the user tracking their blood pressure level.

This device features a built-in "Bluetooth® Data Transmission" function, which enables the device to automatically transmit measuring results to a paired Bluetooth-enabled device. This function allows users to synchronize to current date and time, and check the battery status on the Homedics Health+ app.

When using your blood pressure monitor, basic precautions should always be followed.

Please read and follow all instructions and warnings before using this product. Save these instructions for future reference.

• Please note that this is a home healthcare product only and this manual and device are not intended to serve as a substitute for the advice of a physician or medical professional.

• This device uses the oscillometric method to measure systolic and diastolic blood pressure, as well as heart rate.

• DO NOT use this device for diagnosis or treatment of any health problem or disease. Measurement results are for reference only. Consult a healthcare professional for interpretation of pressure measurements. Contact your physician if you have or suspect any medical problem. Do not change your medications without the advice of your physician or healthcare professional.

• Proper cuff size is critical for accurate measurements. Follow the instructions in this manual and printed on the arm cuff to ensure the arm cuff is used properly.

• If you are pregnant, you should pay more attention to your blood pressure changes, because during this time it may change drastically.

• This monitor is clinically validated for use in pregnancy and preeclampsia. When you detect unusual readings in pregnancy, you should measure again after taking some rest. If the reading is still abnormal, consult your doctor or gynecologist.

• This product is not suitable for:

- People with arrhythmias
- People undergoing intravenous injection on any limb
- People currently in dialysis treatment

• Those who have had a mastectomy surgery (especially when lymph nodes have been removed), it is recommended to take measurements on the unaffected side.

• This device may have difficulty determining proper blood pressure for users with irregular heartbeat, diabetes, liver disease, kidney disease, poor circulation of the blood or for users who have suffered a stroke. Please consult your healthcare professional before using this device.

• Excessive use may result in blood overflow interference, which is likely to cause uncomfortable sensations, such as partial subcutaneous hemorrhage, or temporary numbness to your arm.

In general, these symptoms should not last long. However, if you continue to experience these sensations, please seek advice from a medical professional.

• When used along with other medical electronic equipment on the same arm, pressurization of the cuff may cause the other devices to temporarily malfunction.

• The pulse display is not suitable for checking the frequency of heart pacemakers.

• Electromagnetic interference: The device contains sensitive electronic components. Avoid strong electrical or electromagnetic fields in the direct vicinity of the device (e.g. mobile telephones, microwave ovens). These may lead to temporary impairment of measurement accuracy.

- Consider the electromagnetic compatibility of the device (ex. power disturbance, radio frequency interference, etc.). Please use this device indoors only.
- The pulse display is not suitable for checking the frequency of heart pacemakers.
- Electromagnetic interference: This device contains sensitive electronic components. Avoid strong electrical or electromagnetic fields in the direct vicinity of the device (e.g. cell phones, microwave ovens). These may lead to temporary impairment of measurement accuracy.
- Consider the electromagnetic compatibility of the device (ex. power disturbance, radio frequency interference etc.). Please use this device indoors only.
- Use blood pressure monitor only for its intended use.
- DO NOT wrap the cuff around body parts other than your arm.
- Not for use by or on persons under the age of 18.
- DO NOT use this device on infants, children, or those who cannot express their own intention.
- DO NOT plug or unplug the adapter power cord with wet hands.
- Please use only the USB to DC power cable included with this monitor or 1.5V AAA alkaline batteries for power supply.
- Please rest for at least 5-10 minutes before taking a measurement.
- To allow your blood vessels to return to the condition prior to taking the measurement, please wait at least 3-5 minutes between measurements. You may need to adjust the wait time according to your personal physiological situation.
- Wait 30-45 minutes before measurement if you've just consumed caffeinated beverages or smoked cigarettes.
- The applied part is the cuff.
- The patient is the intended operator.

Blood pressure measurements determined with this device are equivalent to those obtained by a trained observer using the cuff/stethoscope auscultatory method, within the accuracy limits prescribed by the American National Standard, Manual, electronic, or automated sphygmomanometers.

In the event that the device needs to be checked for calibration, contact the distributor.

ABOUT BLOOD PRESSURE

What is blood pressure?
Blood pressure is the measurement of the force of blood pushing against the walls of the arteries. Arterial blood pressure is constantly fluctuating during the course of the cardiac cycle. The pressure measures when the heart contracts and sends blood out of the heart is systolic (highest) blood pressure. The pressure measured when the heart dilates with blood flowing back into the heart is diastolic (lowest) blood pressure.

Why measure your blood pressure?
Among today's various health problems, those associated with high blood pressure are very common. High blood pressure dangerously correlates with cardiovascular diseases. Therefore, blood pressure monitoring is important for identifying those at risk.

Why do my readings vary?
Blood pressure is a body parameter that is subject to normal variations throughout the day. A single reading that is different from yours or your doctor's readings are not necessarily inaccurate. The average of several readings, taken under similar conditions, using the same arm is preferred for accurate blood pressure readings.

Why are my readings different than those taken at my doctor's office?
Having one's blood pressure measured by a doctor in a hospital or a clinic is often associated with an event called "White Coat Hypertension" where the patient becomes nervous or anxious, thus raising his blood pressure. There are also numerous other factors that might cause your blood pressure to be raised at a specific time of day. This is why medical physician recommend home monitoring as it is important to get readings of blood pressure during different times of the day to really get an idea of your real blood pressure.

BLOOD PRESSURE STANDARD

The table below contains defined levels for hypertension that are publicly available from the American Heart Association® (AHA 2017). Users can compare their own blood pressure readings against these defined levels to determine if they may be potentially at increased risk.

Blood Pressure Category	Systolic mm Hg (upper number)	Diastolic mm Hg (lower number)	
Normal	Less than 120	and	Less than 80
Elevated	120–129	and	Less than 80
High Blood Pressure (hypertension) Stage 1	130–139	or	80–89
High Blood Pressure (hypertension) Stage 2	140 or Higher	or	90 or Higher
Hypertension Crisis (consult your doctor immediately)	Higher than 180	and/or	Higher than 120

*Source: American Heart Association® (AHA) 2017

Blood pressure tends to go up and down, even in people who normally don't have high readings. Individual blood pressures vary greatly both on a daily and a seasonal basis. These variations are even more pronounced in hypertensive patients. Normally the blood pressure rises while at work and is at its lowest during sleeping period. If your numbers stay above the "normal" range most of the time, you may be at increased risk and should consult your physician.

Although one can easily find where their own blood pressure readings fall on this table, this monitor comes equipped with a Risk Category Index that automatically compares each reading to the defined levels and provides a helpful cue if your reading falls into one of the stages that could potentially indicate increased risk. See Risk Category Index section for more information on this feature.

HOW THIS BLOOD PRESSURE MONITOR WORKS

This monitor uses Smart Measure® inflation technology to detect your blood pressure. Once the cuff is wrapped around your upper arm, with a press of the START/STOP button, the monitor will turn on and the inflation will automatically start, creating pressure around the arteries inside the upper arm.

Within the cuff is a gauge which senses the fluctuations (oscillations) in pressure. The fluctuation measured represents the degree of intensity that your arteries are contracting with each heartbeat, which is also a result of the pressure that the cuff has placed on the upper arm. The monitor measures these contractions and converts the information to a digital value. Once the measurement is complete, the cuff will automatically deflate, and the result is displayed on the screen.


Please note that any muscle movement during inflation will cause measurement error. When measurement is complete, the monitor will display your systolic pressure, diastolic pressure, and pulse readings. Before measurement, it is suggested that you sit quietly for 15 minutes before measurement as measurements taken in a relaxed state have greater accuracy.

The monitor automatically finds where your measurement results fall on the American Heart Association® (AHA 2017) table and provides a cue if your reading falls into one of the stages that could potentially indicate increased risk. Please refer to the Risk Category Index section for more information on this feature.

The appearance of the icon indicates that a pulse irregularity consistent with an irregular heartbeat was detected during measurement. Please refer to the Irregular Heartbeat Detector section for more information on this feature.

*Source: American Heart Association® (AHA) 2017

NAME/FUNCTION OF EACH PART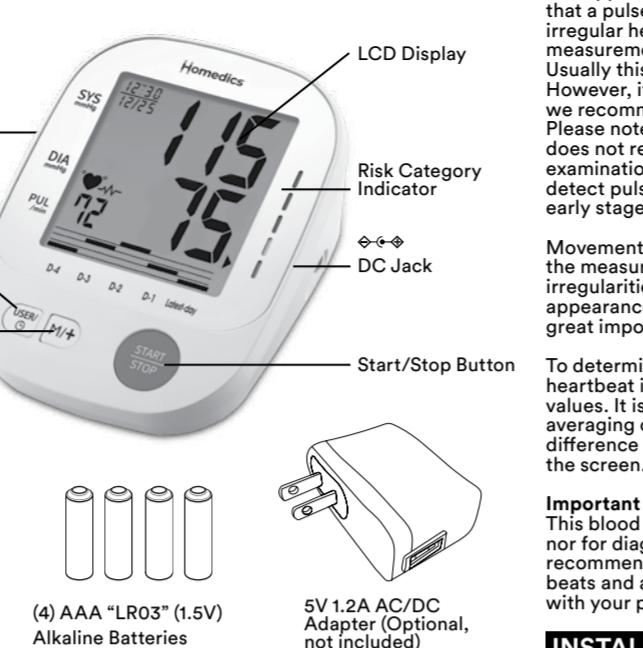

IRREGULAR HEARTBEAT DETECTOR (IHB)

The appearance of the icon indicates that a pulse irregularity consistent with an irregular heartbeat was detected during measurement. Usually, this is not a cause for concern. However, if the symbol appears often, we recommend you seek medical advice. Please note that the device does not replace a cardiac irregular examination, but serves to detect pulse irregularities at an early stage.

Movement, shaking or talking during the measurement can result in pulse irregularities that may cause the appearance of this icon. Therefore, it is of great importance to move or talk during measurement.

To determine the presence of an irregular heartbeat, the average of the heartbeat intervals is calculated with the first 3 normal effective heartbeat values. It is important to note that the average is not a strict mathematical averaging of all recorded intervals. At least 3 beats with 25% or greater difference from the average heartbeat interval will generate the icon on the screen.


Important Information:
This blood pressure monitor is not designed for use by people with arrhythmias nor for diagnosing or treating an arrhythmia problem. As a safeguard, we recommend that if you have arrhythmias such as atrial or ventricular premature beats and atrial fibrillation or any other special conditions you should check with your physician before using your blood pressure monitor.

INSTALLING BATTERIES

1. Slide the battery cover and lift the cover to open the battery compartment.
2. Install or replace 4 AAA (1.5V, LRO3) sized alkaline batteries in the battery compartment according to the indications inside the compartment.
3. Close the battery cover by sliding the cover back into place.

Replace the batteries if:

- The low battery symbol appears.
- When any button is pressed and nothing is displayed on the screen.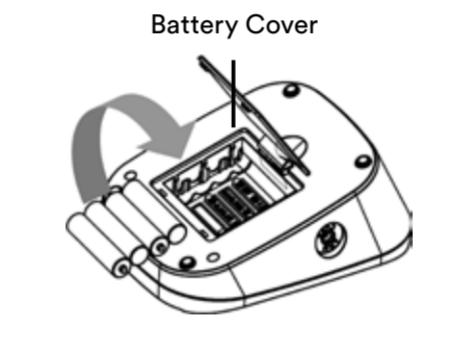

NOTE:

- If the batteries are removed or replaced, the date and time will need to be reset either manually or automatically using your Bluetooth® mobile device.
- Replace all batteries at one time (as simultaneous set). Use only 1.5V AAA alkaline batteries.
- DO NOT mix alkaline, standard (carbon-zinc) or rechargeable (cadmium) batteries.
- When the batteries are removed, the measurement values stored in memory are retained. However, the date and time must be reset.
- Remove batteries when unit is not in use for extended periods of time.
- Clean contacts on battery and in battery compartment with a soft dry cloth.
- Remove depleted batteries from the unit promptly. Dispose of used batteries safely.
- Batteries are hazardous waste. Do not dispose of them together with the household garbage.
- DO NOT dispose of batteries in fire. Batteries may explode or leak.
- Recycle or dispose of properly in accordance with local, state, province, and country regulations.
- Keep batteries away from children.
- Batteries are small objects and could be ingested. If swallowed, contact a physician at once.

DISPLAY SYMBOL EXPLANATIONS

Display Symbol Explanations:

- User 1: Appears when the monitor is operated by User 1.
- User 2: Appears when the monitor is operated by User 2.
- Low Battery Symbol: Appears when batteries should be replaced.
- Pulse Symbol: Once pulse is detected, the symbol flashes with each pulse beat. Shows the heart rate per minute.
- Risk Category Index: The arrow points out to the specific Risk Category that your measurement reading falls in. See Risk Category Index Section for more information.
- Irregular Heartbeat Detector: This symbol appears when an irregular heart beat was detected. See Irregular Heartbeat Detector section for more information.
- Excessive Body Motion Detector: Displayed if body movement is detected during measurement. Especially when the blood pressure monitor is worn on the arm, the blood pressure monitor is worn on, if the cuff is worn improperly, or the shape of the upper arm is unusual (for example, the circumference of the upper arm differs largely from the circumference of the forearm), excessive gap might exist between the arm cuff and the arm.
- Bluetooth® Symbol: Appears when Bluetooth is active, and when data is transmitting to your mobile device.
- AVG. 3: The Average of the Last 3 Measurements: This symbol appears when the LCD displays average value of last 3 readings.
- 5-Day Trend Manage Function Bar Graph: This symbol appears when a measurement is completed or a memory is recalled. It can plot the trend of measurement records in five days. Notice: Please set the correct Date/Time and User-Number before each measurement.

If any of the following letters and numbers appear in the area that systolic pressure should be displayed, an error has occurred with your reading. See Troubleshooting section of this manual for more information.

EE Measurement Error: Wrap the cuff correctly and keep arm steady during measurement. Take new measurement.

E1 Air Circuit Abnormality: Check cuff connection. Take new measurement.

E2 Pressure Exceeding 300 mmHg: Turn the unit off to clear, then take new measurement.

E3 Error Determining Measurement Data: Ensure the cuff is worn properly, do not move or talk. Take new measurement.

E4 Data Transmission Error: Will occur after 45 seconds, if the monitor cannot connect to the mobile device to transmit data. Make sure Bluetooth® is ON.

EP System Error: Contact Consumer Relations.

Use only a compatible USB/DC or AC/DC adapter with required voltage and current as indicated in this manual.

• Recommend Adapter specification, do not use otherwise:
Model: FranMar International, FRM06-S05-UU

• Rating:
Input: 100-240V, 50/60 Hz, 0.2 A
Output: 5V, DC, 1.2A,

NOTE:

- When the USB to DC cable, or AC adapter is your main power supply, make sure the adapter plug can be easily removed from the unit.
- Please unload the batteries when operating with an USB to DC cable, or AC adapter for an extended period of time. Leaving the batteries in the compartment for a long time may cause leakage, which may lead to damage to the unit.
- No batteries are needed when operating with an USB to DC cable, or AC adapter.
- Date and time will need to be reset if USB to DC cable, or AC adapter is unplugged even if the unit has the batteries in the battery compartment.

DATE & TIME SET PROCEDURE

Date & time can be set by two methods, either sync automatically using your Bluetooth® mobile device, or set manually using the User/Date & Time set and Memory/Date & Time Select buttons on the monitor.

If setting the date & time by syncing with your mobile device, it is important that this is done prior to taking any measurements to ensure the date & time are accurate.

To set using your device with Bluetooth® wireless technology:

1. Before taking the first measurement, open the Homedics Health+ app on your device. If it is not already installed, download it from the App Store and/or Google Play®. Make sure it is fully downloaded and open on your mobile device before trying to sync the Date & Time. Make sure the Bluetooth® is ON on your mobile device and your App is open.
2. Follow the prompts in the app to select your device, sync to the app, and auto set the date & time.

Note:

- Once the date & time have been successfully synced, future readings will automatically have the correct date & time.
- If you are having trouble automatically syncing the date & time, make sure the Bluetooth® is ON on your device and ON on the blood pressure monitor.

To Set Manually:

1. To set the date and time, the monitor needs to be in standby mode. Press START/STOP button and the monitor will automatically turn to standby mode. Then press User/Date & Time set button.

2. The display will show a blinking number showing the YEAR. Change the YEAR by pressing the Memory/Date & Time Select button. Each press will increase the number by 1 in a rolling manner. Press the User/Date & Time set button again to confirm the entry, and the screen will show a blinking number representing the MONTH.


3. Change the DAY, HOUR and MINUTE and select OFF, or leave ON Bluetooth function as described above, using the Memory/Date & Time Select button to change the numbers and the User/Date & Time set button to confirm the entries.

4. The date and time will only need to be set manually if the monitor will not be used with the Homedics Health+ app.

5. If the User

TRANSMIT READINGS TO YOUR MOBILE DEVICE

This monitor syncs your blood pressure readings to the Homedics Health+ app that is free to download on the App Store® and on Google Play™. Make sure you have the Homedics Health+ app downloaded, and open on your mobile device before trying to transmit your blood pressure measurements.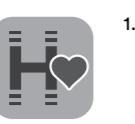

Before attempting to sync the blood pressure monitor with your mobile device, make sure Bluetooth® is turned ON, on both your mobile device and the monitor.

Transmit Reading

There are 2 ways to activate the Bluetooth® function.

- Automatically activate:** When measurement completed, the device activates the Bluetooth® function automatically, and Bluetooth® symbol will flash on the screen.
- NOTE:** The app must be open for the data to transmit successfully.

- Manually activate:** Under standby mode, user can press the button for 3 seconds to activate the Bluetooth® function, and the Bluetooth symbol will flash on the screen.
- NOTE:** The app must be open for the data to transmit successfully.

MEASUREMENT PROCEDURE

Important Note:

- Blood pressure changes with every heartbeat and is in constant fluctuation throughout the day.
- Blood pressure measurement can be affected by the position of the user, his or her physiologic condition and other factors.

Before Measurement:

To help ensure an accurate reading, follow these instructions before taking a measurement:

- Wait 30 minutes after exercising, bathing, eating, drinking beverages with alcohol or caffeine, or smoking to measure blood pressure.
- Sit quietly and rest for 5 minutes.
- Empty your bladder, as a full one can temporarily raise blood pressure.
- Stress raises blood pressure. Avoid taking measurements during stressful times.
- Take your reading in a comfortable environment as measurements can be affected by hot or cold temperatures. Take your blood pressure at normal body temperature.



During Measurement:

- DO NOT talk or move your arm or hand muscles.
- DO NOT cross your legs. Sit with feet flat on the floor.
- DO NOT touch cuff or monitor during measurement.

If you are using this blood pressure monitor for the first time, please remove the protective film from the screen.

Taking a Measurement:

- Under Standby Mode, press User/Date & Time set button to select User 1 or 2.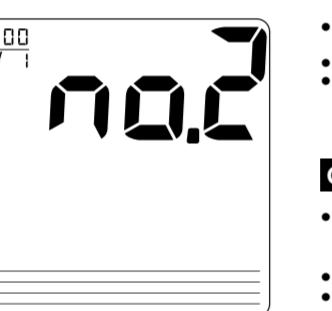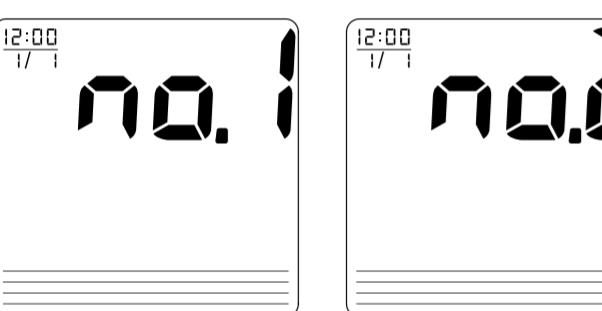

- With the cuff wrapped around your arm, press the START/STOP button to confirm the chosen user and start measurement.

- All display symbols appear on the screen for 1.5 seconds. After all symbols disappear, the display will show "00". The monitor is "Ready to Measure" and will automatically inflate to the level that is right for you.

- As the cuff inflates, the monitor automatically determines your ideal inflation level. This monitor features Smart Measure® inflation technology, and detects your blood pressure and pulse rate during inflation. The Heartbeat Symbol flashes at every heartbeat. Remain still and do not move until the entire measurement process is completed. The device will detect your pulse and determine the blood pressure.

NOTE:
If the cuff does not stop inflating, remove the cuff at once.
To stop measurement, press START/STOP button. The cuff will deflate immediately after the button is pressed.

- After the monitor has determined your blood pressure and heart rate, the cuff automatically deflates. Your results including systolic pressure, diastolic pressure, heart rate, corresponding Risk Category Indicator, graphic of 5 Day Trend Function, Irregular Heartbeat Detector and Excessive Body Motion Detector (if any) will be displayed with date and time for 1 minute and saved to memory automatically.

- After measurement is completed, the device activates Bluetooth function automatically and the Bluetooth symbol will flash on the screen. If the monitor cannot be connected to paired Bluetooth-enabled device over 45 seconds, LCD will display Error message "E4" and Bluetooth will be turned off.

- Device automatically shuts off if no operation over 1 minute.



USING THE MEMORY FUNCTION

The monitor can store up to 120 memory sets each for 2 users, and automatically replace the oldest data with a new one.


After each measurement, the results including systolic and diastolic pressure, heart rate and corresponding Risk Category Indicator, graphic of 5-Day Trend Function, Irregular Heartbeat Detector and Excessive Body Motion Detector (if any) with the time and date will be automatically stored. The monitor features 2 user memory capabilities. The monitor holds the last 120 measurements for each user, and automatically replaces the oldest data with a new one.

RECALLING VALUES FROM MEMORY

- Press the User/Date & Time set button to select User 1 or User 2.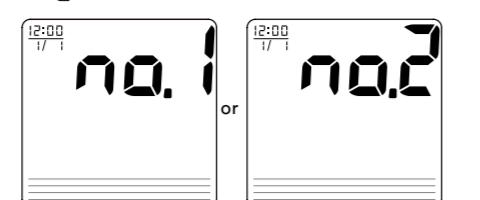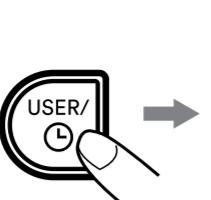

- Press the Memory/Date & Time Select button to access memory. If there is no data stored before, nothing (except month, date and time) will appear on the display. If yes, the first reading will be the average of the last 3 measurements.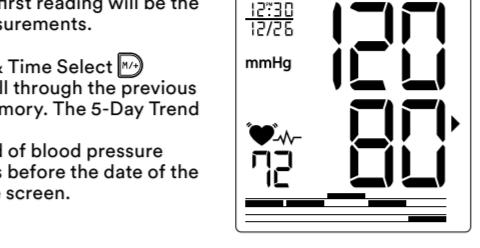

- Press the Memory/Date & Time Select button repeatedly to scroll through the previous readings stored in the memory. The 5-Day Trend bar graph will plot the "5-Day" trend of blood pressure variation, spanning 4 days before the date of the reading that shows on the screen.



NOTE:
For viewing the 5-Day Trend bar graph:

- The rightmost bar, "Latest day", will always correspond with the date of reading that shows on the screen.
- For more detailed explanation about the 5-Day Trend Manage function, please see the "5-Day Trend" section.

- To stop reading the memories, press the START/STOP button to switch to Standby Mode.

CLEARING VALUES FROM MEMORY

- Press the User/Date & Time set button to select User 1 or User 2.
- Press the Memory/Date & Time Select button to access the memory recall mode.

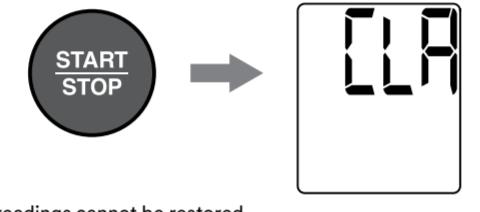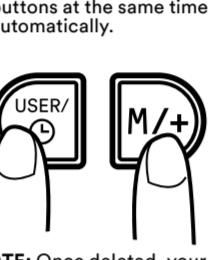NOTE: Once deleted, your readings cannot be restored.

IMPORTANT NOTES REGARDING YOUR BLOOD PRESSURE MEASUREMENT

- It is suggested that you take your measurements at the same time each day and use the same arm for consistency.
- Users should wait a minimum of 5 minutes before taking additional measurements. More time may be necessary depending upon your physiology. The measurement results that users receive are for reference only. If users have any blood pressure concerns, please consult a physician.
- Once a measurement reaches 300 mmHg, the unit will deflate automatically for safety reasons.
- This device is not suitable for people with arrhythmias.
- This device may have difficulty determining the proper blood pressure for users with irregular heartbeat, diabetes, poor circulation of blood, kidney problems, or for users who have suffered a stroke.

CARE, MAINTENANCE, & CLEANING

- Clean the blood pressure monitor body and cuff carefully with a slightly damp, soft cloth. DO NOT press. Do not wash cuff or use chemical cleaner on it. Never use thinner, alcohol or petrol (gasoline) as cleaner.
- Make sure the cuff is completely dry before using.

- Leaky batteries can damage the unit. Remove the batteries when the unit will not be used for a long time.
- Follow local ordinances and recycling instructions regarding disposal or recycling of the device and device components, including batteries.

- If the unit is stored near freezing, allow it to acclimate to room temperature before use.

- This blood pressure monitor is not field serviceable. You should not use any tool to open the device nor should you attempt to adjust anything inside the device. If you have any problems with this device, please contact Homedics Consumer Relations (contact information on warranty page).

- DO NOT subject the unit in water as this will result in damage to the unit.

- DO NOT subject the monitor or cuff to extreme temperatures, humidity, moisture, or direct sunlight. Protect from dust.

- DO NOT fold the cuff tightly.

- DO NOT disassemble the monitor or cuff. If in need of repair, refer to the warranty section of this manual.

- DO NOT subject the monitor to extreme shocks (DO NOT drop on floor).

- DO NOT twist the cuff in any way.

- DO NOT inflate the cuff unless wrapped around arm.

- DO NOT wrap the cuff around body parts other than your arm.

- DO NOT drop or insert any object into any opening or hose.

- Avoid accidental strangulation keep this product away from children. Do not drape tube around neck.

- This monitor may not meet its performance specifications if stored or used outside of these temperature and humidity ranges:

POTENTIAL FOR ELECTROMAGNETIC INTERFERENCE

To avoid inaccurate results caused by electromagnetic interference between electrical and electronic equipment, do not use the device near a cell phone or microwave oven. For most wireless communication devices, it is recommended to maintain a distance of 10.8 feet (3.3m) in order to avoid electromagnetic interference.

SUPPLIER'S DECLARATION OF CONFORMITY

Product Description: Blood Pressure Monitor

Model Number: BPA-100TRBT, BPA-100TRBT

Trade Name: Homedics

FCC Compliance Statement

This device complies with Part 15 of the FCC Rules. This device contains license-exempt transmitters/receivers that comply with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) This device must accept any interference received, including interference that may cause undesired operation.

U.S. Contact Information

Company: Homedics, LLC.

Address: 3000 N. Pontiac Trail, Commerce Township, MI 48390

8:30am-7:00pm EST Monday-Friday 1-800-466-3342

Homedics is not responsible for any radio or TV interference caused by unauthorized modifications to this equipment. Such modifications could void the user authority to operate the equipment, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.

- Increase the separation between the equipment and receiver.

- Connect the equipment to an outlet on a circuit different from that to which the receiver is connected.

- Consult the dealer or an experienced radio/TV technician for help.

CAN ICES-003(B)/NMB-003(B)

RF Exposure Statement

This equipment complies with FCC and ICED RF radiation exposure limits set forth for an uncontrolled environment. This transmitter must not be co-located or operated in conjunction with any other antenna or transmitter.

Test specifications for enclosure port immunity to RF wireless communications equipment.

ELECTROMAGNETIC COMPATIBILITY (EMC)

Guidance and manufacturer's declaration – electromagnetic emissions
The device is intended for use in the electromagnetic environments listed below, and should only be used in such environments:

Emissions Test	Compliance	Electromagnetic Environment - Guidance
RF emissions CISPR 11	Group 1	RF energy is used only to maintain device's operation. Therefore, its RF emissions are so low that it's unlikely to cause any interference in nearby.
RF emissions CISPR 11	Class B	
Harmonic Emissions IEC 61000-3-2	Class A	The device is suitable for use in all establishments, including domestic establishments, and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Complies	

Guidance and manufacturer's declaration – electromagnetic immunity
The device is intended for use in the electromagnetic environments listed below, and should only be used in such environments:

Immunity Test	IEC 60601 Test Level	Compliance Level	Electromagnetic Environment-Guidance
Electrostatic discharge (ESD) IEC 61000-4-2	$\pm 8\text{ kV}$ contact discharge $\pm 15\text{ kV}$ air discharge	$\pm 8\text{ kV}$ contact $\pm 15\text{ kV}$ air discharge	In the case of air discharge testing, the climatic conditions shall be met in the following ranges: Ambient Temperature: $15^\circ\text{C} \sim 35^\circ\text{C}$; Relative Humidity: 30% ~ 60%.
Power frequency (50 or 60 Hz) magnetic field IEC 61000-4-8	30 A/m 50 or 60 Hz	30 A/m 50 or 60 Hz	Power frequency magnetic fields shall be at levels characteristic of a typical location in a typical commercial or hospital environment.
Electrical fast transient/burst IEC 61003-4-4	$\pm 2\text{ kV}$ for power supply lines $\pm 2\text{ kV}$ for input/output lines	$\pm 2\text{ kV}$ for power supply lines $\pm 2\text{ kV}$ for input/output lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	AC Power port: $\pm 1\text{ KV Line to Line}$	AC Power port: $\pm 1\text{ KV Line to Line}$	Mains power quality should be that of a typical commercial or hospital environment.
Interruptions and voltage variations on power supply input lines IEC 61003-4-11	OR6 UT: 0.5 cycle At $0^\circ, 45^\circ, 90^\circ, 135^\circ, 180^\circ, 225^\circ, 270^\circ$ and 315° OR6 UT: 1 cycles $70\% UT; 25/30 cycles$ OR6 UT: 0.5 cycle $70\% UT; 250/300 cycle$	OR6 UT: 0.5 cycle At $0^\circ, 45^\circ, 90^\circ, 135^\circ, 180^\circ, 225^\circ, 270^\circ$ and 315° OR6 UT: 1 cycles $70\% UT; 25/30 cycles$ OR6 UT: 0.5 cycle $70\% UT; 250/300 cycle$	Mains power quality should be that of a typical commercial or hospital environment. If the user of the device requires continued operation during power mains interruptions, it is recommended that the device be powered from an uninterruptible power supply or a battery.

Guidance and manufacturer's declaration – electromagnetic immunity

The device is intended for use in the electromagnetic environments listed below, and should only be used in such environments:

Immunity Test	IEC 60601 Test Level	Compliance Level	Electromagnetic Environment-Guidance
---------------	----------------------	------------------	--------------------------------------

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