# Homedics®

luxe compression  
foot & calf massager

**please take a  
moment now**  
register your product at  
[www.Homedics.com/register](http://www.Homedics.com/register)  
Your valuable input regarding this  
product will help us create  
the products you will want  
in the future.

**INSTRUCTION MANUAL AND WARRANTY INFORMATION**

**2-year limited warranty**

FMS-502H | L-03685, Rev. 2

# IMPORTANT SAFETY INSTRUCTIONS

WHEN USING ELECTRICAL PRODUCTS, ESPECIALLY WHEN CHILDREN ARE PRESENT, BASIC SAFETY PRECAUTIONS SHOULD ALWAYS BE FOLLOWED, INCLUDING THE FOLLOWING:

READ ALL INSTRUCTIONS BEFORE USING.

## **DANGER** – TO REDUCE THE RISK OF ELECTRIC SHOCK:

- Always unplug this appliance from the electrical outlet immediately after using and before cleaning.
- DO NOT reach for a unit that has fallen into water. Unplug it immediately.

## **WARNING** – TO REDUCE THE RISK OF BURNS, FIRE, ELECTRIC SHOCK, OR INJURY TO PERSONS:

- An appliance should never be left unattended when plugged in. Unplug from outlet when not in use, and before putting on or taking off parts or attachments.
- Close supervision is necessary when this appliance is used by, on, or near children, invalids, or disabled persons.
- DO NOT use outdoors.
- Use this appliance only for its intended use as described in this manual. Do not use attachments not recommended by Homedics; specifically, any attachments not provided with the unit.
- DO NOT carry this appliance by the supply cord or use cord as a handle.
- NEVER operate this appliance if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or if it has been dropped into water. Return the appliance to the Homedics Service Center for examination and repair.
- Keep the cord away from heated surfaces.
- NEVER operate the appliance with the air openings blocked. Keep the air openings free of lint, hair, and the like.
- NEVER drop or insert any object into any opening.
- DO NOT operate where aerosol (spray) products are being used or where oxygen is being administered.
- DO NOT operate under a blanket or pillow. Excessive heating can occur and cause fire, electric shock, or injury to persons.
- To disconnect, turn all controls to the off position, then remove plug from outlet.
- DO NOT stand on or in appliance. Use only while seated.
- Keep long hair away from massager while in use.

- DO NOT use massager in close proximity to loose clothing or jewelry.
- Use heated surfaces carefully. May cause serious burns. Do not use over insensitive skin areas or in the presence of poor circulation. The unattended use of heat by children or incapacitated persons may be dangerous.

## **SAVE THESE INSTRUCTIONS**

### **CAUTION** – PLEASE READ ALL INSTRUCTIONS CAREFULLY BEFORE OPERATING.

- This product is not intended for medical use.
- Consult your doctor prior to using this product, if:
  - You are pregnant
  - You have a pacemaker
  - You have any concerns regarding your health
- Not recommended for use by diabetics.
- NEVER leave the unit unattended, especially if children are present.
- NEVER cover the unit when it is in operation.
- DO NOT use this product for more than 15 minutes at a time.
- Extensive use could lead to the product's excessive heating and shorter life. Should this occur, discontinue use and allow the unit to cool before operating.
- NEVER use this product directly on swollen or inflamed areas or skin eruptions.
- DO NOT use this product as a substitute for medical attention.
- NEVER use this product while in bed.
- This product should never be used by any individual suffering from any physical ailment that would limit the user's capacity to operate the controls.
- Not for use by children.
- This unit should not be used by invalids without adult supervision.
- This appliance is intended for household use only.
- This product should NEVER be used by any individual suffering from any physical ailment that would limit the user's capacity to operate the controls or who has sensory deficiencies.

**NOTE:** This unit may turn off after an extended period of use. If your unit turns off, LED may remain on. If your unit overheats, unplug the unit and let cool for 30 minutes before plugging in again and continuing use.

# INSTRUCTIONS FOR USE

To reduce the risk of electric shock, this unit is equipped with a polarized plug (one blade wider than the other). This plug will fit in a polarized outlet only one way. If the plug does not fit fully in the outlet, reverse the plug. If it still does not fit, contact a qualified electrician to install the proper outlet. Do not change the plug in any way.

1. Plug into a 120-volt outlet.
2. DO NOT use the unit with shoes.
3. Place massager on the floor in front of your chair or seat.
4. See pages 6–7 for control panel instructions.
5. Unzip the compression boots, insert your feet into the massager, and rezip the boots.

**NOTE: For your safety, there is a 15-minute auto shutoff on the unit.**

## FEATURES

### **Soothing Heat**

Adds to your comfort and relaxation.

### **2 Auto Programs**

Deliver a preset sequence of massage features and intensities. Program 1 is designed to soothe and relax. Program 2 is designed to energize and promote muscle recovery.

### **Customized Controls**

Allow you to select massage programs and intensity levels.

### **Zippered Boot**

Fits small and large calf sizes.

# **MAINTENANCE**

## **TO CLEAN THE MASSAGER**

Unplug the unit and allow it to cool before cleaning. Clean only with a soft, slightly damp sponge. Never allow water or any other liquids to come into contact with the unit.

- DO NOT immerse in any liquid to clean.
- NEVER use abrasive cleaners, brushes, or strong chemicals that may or may not be flammable and/or damaging to the product to clean.
- DO NOT attempt to repair the massager. There are no user-serviceable parts. For service, contact Consumer Relations at the telephone number listed in the Warranty section.

## **TO STORE**

Place massager in its box or in a cool, dry place. Avoid contact with sharp edges or pointed objects which might cut or puncture the fabric surface. To avoid breakage, do not wrap the power cord around the unit. Do not hang the unit by the power cord.

---

**CAUTION: All servicing of this massager must be performed by authorized Homedics service personnel only.**

## **POWER**

Press to power unit on and off. When the unit is turned on it will start in the auto program 1 by default.

## **AIR COMPRESSION INTENSITY**

Press once for Low intensity – white led


Press again for Medium intensity – blue led

Press a third time for High intensity – green led

Pressing a fourth time will return to low intensity.

## **KNEADING SOLE MASSAGE**

Press once to turn on the kneading massager under your feet; the button will illuminate white. Press again to turn the kneading off.

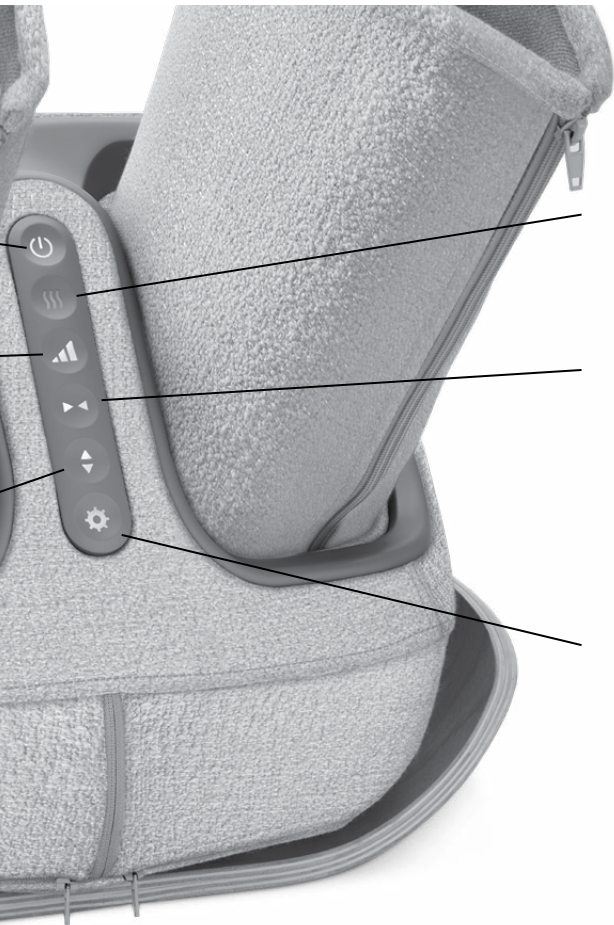### **HEAT**

Press to turn heat on/off. The button illuminates red when heat is on.

### **AIR COMPRESSION MASSAGE**

Press once for air compression in leg and foot. Press again for air compression in foot only. Press a third time for air compression in leg only. Press a fourth time to turn off air compression.

### **AUTO PROGRAMS**

There are 2 auto programs. When the unit is powered on, it will start in auto program 1 by default. This button will illuminate white when in auto program.

1. Press the button a second time to change to auto program 2. The button will illuminate blue. To exit the auto program mode, press the Air Compression or the Kneading button.

# SUPPLIER'S DECLARATION OF CONFORMITY

**Product Description:** Luxe Compression Foot & Calf Massager

**Model Number:** FMS-502H

**Trade Name:** Homedics

## FCC Compliance Statement

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) This device must accept any interference received, including interference that may cause undesired operation.

## U.S. Contact Information

**Company:** Homedics, LLC.

**Address:** 3000 N Pontiac Trail, Commerce Township, MI 48390

**8:30am-7:00pm EST Monday-Friday 1-800-466-3342**

Homedics is not responsible for any radio or TV interference caused by unauthorized modifications to this equipment. Such modifications could void the user authority to operate the equipment.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment to an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

CAN ICES-003(B)/NMB-003(B)

## 2-YEAR LIMITED WARRANTY

Homedics sells its products with the intent that they are free of defects in manufacture and workmanship for a period of 2 years from the date of original purchase, except as noted below. Homedics warrants that its products will be free of defects in material and workmanship under normal use and service. This warranty extends only to consumers and does not extend to Retailers.

To obtain warranty service on your Homedics product, contact a Consumer Relations representative for assistance. Please make sure to have the model number of the product available.

Homedics does not authorize anyone, including, but not limited to, Retailers, the subsequent consumer purchaser of the product from a Retailer or remote purchasers, to obligate Homedics in any way beyond the terms set forth herein. This warranty does not cover damage caused by misuse or abuse; accident; the attachment of any unauthorized accessory; alteration to the product; improper installation; unauthorized repairs or modifications; improper use of electrical/power supply; loss of power; dropped product; malfunction or damage of an operating part from failure to provide manufacturer's recommended maintenance; transportation damage; theft; neglect; vandalism; or environmental conditions; loss of use during the period the product is at a repair facility or otherwise awaiting parts or repair; or any other conditions whatsoever that are beyond the control of Homedics.

This warranty is effective only if the product is purchased and operated in the country in which the product is purchased. A product that requires modifications or adoption to enable it to operate in any other country than the country for which it was designed, manufactured, approved and/or authorized, or repair of products damaged by these modifications is not covered under this warranty.

THE WARRANTY PROVIDED HEREIN SHALL BE THE SOLE AND EXCLUSIVE WARRANTY. THERE SHALL BE NO OTHER WARRANTIES EXPRESS OR IMPLIED INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS OR ANY OTHER OBLIGATION ON THE PART OF THE COMPANY WITH RESPECT TO PRODUCTS COVERED BY THIS WARRANTY. HOMEDICS SHALL HAVE NO LIABILITY FOR ANY INCIDENTAL, CONSEQUENTIAL, OR SPECIAL DAMAGES. IN NO EVENT SHALL THIS WARRANTY REQUIRE MORE THAN THE REPAIR OR REPLACEMENT OF ANY PART OR PARTS WHICH ARE FOUND TO BE DEFECTIVE WITHIN THE EFFECTIVE PERIOD OF THE WARRANTY. NO REFUNDS WILL BE GIVEN. IF REPLACEMENT PARTS FOR DEFECTIVE MATERIALS ARE NOT AVAILABLE, HOMEDICS RESERVES THE RIGHT TO MAKE PRODUCT SUBSTITUTIONS IN LIEU OF REPAIR OR REPLACEMENT.

This warranty does not extend to the purchase of opened, used, repaired, repackaged and/or resealed products, including but not limited to sale of such products on internet auction sites and/or sales of such products by surplus or bulk resellers. Any and all warranties or guarantees shall immediately cease and terminate as to any products or parts thereof which are repaired, replaced, altered, or modified, without the prior express and written consent of Homedics.

This warranty provides you with specific legal rights. You may have additional rights which may vary from state to state and country to country. Because of individual state and country regulations, some of the above limitations and exclusions may not apply to you.

For more information regarding our product line in the USA, please visit [www.Homedics.com](http://www.Homedics.com). For Canada, please visit [www.Homedics.ca](http://www.Homedics.ca).

---

**FOR SERVICE IN THE USA**  
CSERVICE@HOMEDICS.COM  
8:30AM-7:00PM EST MONDAY-FRIDAY  
1-800-466-3342

**FOR SERVICE IN CANADA**  
CSERVICE@HOMEDICSGROUP.CA  
8:30AM-5:00PM EST MONDAY-FRIDAY  
1-888-225-7378

©2023 Homedics, LLC. All rights reserved.  
Homedics is a registered trademark of Homedics, LLC.  
Distributed by Homedics, LLC, 3000 N Pontiac Trail, Commerce Township, MI 48390  
IB-FMS502H  
Made in China

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©2023 Homedics, LLC. Todos los derechos reservados.
Homedics es una marca registrada de Homedics, LLC.
Distribuido por Homedics, LLC, 3000 N Pontiac Trail, Commerce Township, MI 48390