Homedics®

theraspa luxe paraffin bath

**please take a
moment now
register your product at**
www.Homedics.com/register

Your valuable input regarding this
product will help us create
the products you will want
in the future.

INSTRUCTION MANUAL AND WARRANTY INFORMATION

2-year limited warranty

PAR-375H

IMPORTANT SAFETY INSTRUCTIONS

WHEN USING ELECTRICAL PRODUCTS, ESPECIALLY WHEN CHILDREN ARE PRESENT, BASIC SAFETY PRECAUTIONS SHOULD ALWAYS BE FOLLOWED, INCLUDING THE FOLLOWING:

READ ALL INSTRUCTIONS BEFORE USING.

DANGER – TO REDUCE THE RISK OF ELECTRIC SHOCK:

- Always unplug the appliance from the electrical outlet immediately after using and before cleaning.
- DO NOT place or store appliance where it can fall or be pulled into a tub or sink.
- DO NOT place in or drop into water or other liquid.
- DO NOT reach for an appliance that has fallen into water. Unplug it immediately.
- DO NOT use while bathing or in a shower.

WARNING – TO REDUCE THE RISK OF BURNS, ELECTRIC SHOCK, FIRE, OR INJURY TO PERSONS:

- An appliance should never be left unattended when plugged in. Unplug from outlet when not in use, and before putting on or taking off parts or attachments.
- Close supervision is necessary when this appliance is used by, on, or near children, invalids, or disabled persons.
- Use this appliance only for its intended use as described in this manual. Do not use attachments not recommended by Homedics; specifically, any attachments not provided with the unit.
- NEVER operate this appliance if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water. Return the appliance to a Homedics Service Center for examination and repair.
- Keep the cord away from heated surfaces.
- NEVER drop or insert any object into any opening or hose.
- DO NOT operate under a blanket or pillow. Excessive heating can occur and cause fire, electric shock, or injury to persons.
- DO NOT use outdoors.
- NEVER operate the appliance with the air openings blocked. Keep the air openings free of lint, hair, and the like.
- This appliance is designed for personal, nonprofessional use.

- NEVER use while sleepy or drowsy.

CAUTION: All servicing of this massager must be performed by authorized Homedics service personnel only.

SAVE THESE INSTRUCTIONS

CAUTION – PLEASE READ ALL INSTRUCTIONS CAREFULLY BEFORE OPERATING.

- This product is not intended for medical use.
- Not recommended for use by diabetics.
- Use bath only when seated. DO NOT stand on or in the tub unit.
- DO NOT apply paraffin wax on open cuts or wounds, warts, inflamed skin, or when the sensation of any extremities is decreased.
- Consult your physician if you have any questions or concerns about acute inflammation and peripheral vascular disease or decreased sensation of any extremity before using.
- Discontinue use and consult a physician if the paraffin wax feels too warm or if it does not feel warm enough. This may indicate a possible physical problem.
- Discontinue use and consult your physician if dermatitis, discomfort, irritation, or pain occurs from continued paraffin treatments.
- Test the paraffin wax on the inside of your wrist to make sure the temperature is comfortable.
- DO NOT overfill the unit and always keep the paraffin wax level about 2 inches from the top of the bath.
- NEVER operate the unit without paraffin wax.
- DO NOT melt paraffin wax in its plastic packaging, nor melt it over an open flame, on a range, stove, or in a microwave.
- DO NOT pour the melted paraffin wax from the tub/unit.
- Unplug unit before filling with paraffin wax.
- DO NOT use accessories or wax not recommended or sold by Homedics.
- If the supply cord is damaged, it must be replaced by a Homedics Service Center.
- DO NOT put water into the tub/unit.
- DO NOT add oil or any other additives to the paraffin wax.
- This appliance is intended for household use only.

HOW TO SET UP THE THERASPA LUXE UNIT:

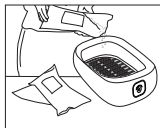
Fig. 1

1. Remove all contents from packaging and check to make sure you have all of the following components of the TheraSpa Luxe paraffin bath:

- Paraffin bath tub with lid
- Plastic grate
- Three 1-lb. bags of paraffin wax
- 20 plastic liners

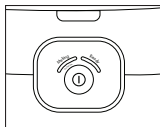
Fig. 2

2. Place unit on a level and sturdy table or surface and open the lid.
3. Make sure TheraSpa Luxe tub is empty before putting wax into tub.
4. Open the bags of paraffin wax and place wax in the tub. Always keep the level of melted paraffin wax filled to the "MIN" level marked on the inside of the tub.

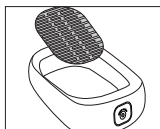
Fig. 3

Note:

To avoid burns, be sure to add the grate before dipping.

Fig. 4

5. Plug the unit into a 120-volt A/C household outlet.
6. Place the lid on the tub. Push the power button and the Melting light will illuminate white while the wax is melting.
7. The paraffin wax should melt in less than two hours. "Ready" light will come on when wax is completely melted. Open the lid and carefully place the grate into the wax and allow it to sink to the bottom of the tub (Fig. 3). The TheraSpa Luxe unit is designed to be used at room temperature. If the wax starts to solidify or flakes form on the top surface, close lid and press power ⏻ button.

NOTE: The temperature of the room can have an effect on the unit's performance. Therefore, we always recommend testing the paraffin temperature on the inside of the wrist area before dipping your entire hand or elbow.

NOTE: NEVER open lid or dip your hand, foot, or elbow while the melting light is on.

NOTE: Place the grate into the unit only when the paraffin wax is melted.

Fig. 5

INSTRUCTIONS FOR USE

This appliance has a polarized plug (one blade is wider than the other). To reduce the risk of electric shock, the plug is intended to fit into a polarized outlet only one way. If the plug does not fit fully into the outlet, reverse the plug. If it still does not fit, contact a qualified electrician. Do not change the plug in any way. This unit is controlled by a thermostat. The unit can hold a maximum of 4 lbs. of paraffin wax for operation. The melting time is less than 2 hours and it is designed to run continuously for 24 hours a day.

For proper application, quickly dip the body area (i.e. hand, foot, or elbow) into the melted paraffin wax and remove immediately.

- Wait a few seconds for wax to solidify.
- This should be repeated 6–10 times.
- Never soak in the wax for more than 1–2 seconds.

IMPORTANT NOTE:

This unit may feel very warm to sensitive skin. However, in order to fully experience the benefits of warm paraffin, the wax must be melted to a temperature between 123–131 degrees Fahrenheit.

If you are very sensitive to heat or feel that this appliance may not be for you, please return this product to the place of purchase, prior to use.

SERVICING OF DOUBLE-INSULATED APPLIANCES:

In a double-insulated appliance, 2 systems of insulation are provided instead of grounding. No grounding means is provided on a double-insulated appliance, nor should a means for grounding be added to the appliance. Servicing a double-insulated appliance requires extreme care and knowledge of the system, and should be done only by qualified service personnel. Replacement parts for a double-insulated appliance must be identical to the parts they replace. A double-insulated appliance is marked with the words "DOUBLE INSULATION" or "DOUBLE INSULATED." The symbol  may also be marked on the appliance.

HOW TO USE THE PARAFFIN BATH:

1. Before immersing your hand or elbow test the paraffin wax on the inside of your wrist to make sure the temperature of the wax is comfortable. Remove all jewelry before dipping (i.e. rings, bracelets, watch).

NOTE: It is suggested you wash the area to be treated and apply lanolin-based hand lotion over the area before dipping it into the paraffin wax. This will allow the wax to be removed easier and will help moisturize the skin.

2. Dip your hand, elbow or foot into the melted paraffin wax and remove immediately. Wait a few seconds until the wax solidifies. Try to avoid moving your hand or elbow while wax is applied.

NOTE: Dipping process is quick. DO NOT leave hand or elbow in the tub. DO NOT soak. Dip quickly in and out.

3. Repeat this step several times until several layers have formed; 6–10 coatings are suggested.
4. For extended warmth, place a plastic liner (provided) over the treated area and twist bag at opening. Keep paraffin wax and liner on treated area for 10–20 minutes for ultimate luxury (Fig.4).
5. Peel off the wax from the treated area. If using a plastic liner, peel the plastic lining from the area being treated. Paraffin wax will peel off with liner (Fig.5).
6. Discard used wax.

NOTE: If you are the only person using the TheraSpa Luxe, the wax can be re-used. Simply put the wax back into the tub to re-melt.

MAINTENANCE

TO CLEAN


Unplug unit and remove grate. Allow paraffin wax to solidify overnight.

1. Push on one end of paraffin wax block. This will tip it up. Lift the block out of the unit and dispose of it. **DO NOT USE SHARP OBJECTS TO REMOVE THE PARAFFIN WAX BLOCK, THIS MAY SCRATCH OR DAMAGE THE TUB.** If the paraffin wax does not remove easily, plug unit in and allow to heat for 10–15 minutes, then remove the paraffin wax block and unplug the unit.
2. Wipe the inside of the unit with a disposable towel to remove residual dirt and paraffin.
3. With a damp, soapy cloth, wipe clean. **DO NOT POUR WATER INTO THE UNIT OR IMMERSE THE UNIT IN WATER.**

TO STORE

Unplug the appliance from the outlet and allow to cool and dry.

- Store in its box or a dry place.
- To avoid breakage, do not wrap the power cord around the unit.
- DO NOT hang the unit by the power cord.
- This appliance is designed for household use only.

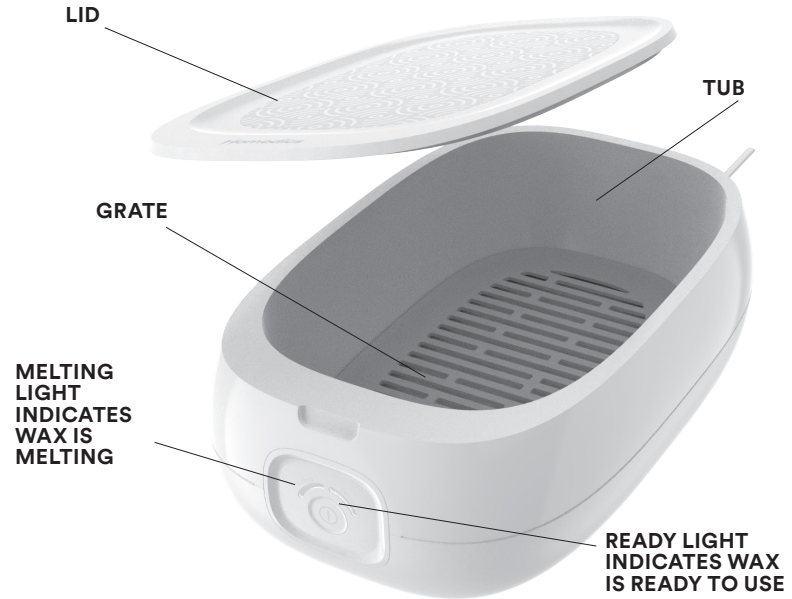20 PLASTIC LINERS

THREE 1-LB. BAGS OF PARAFFIN WAX

Homedics

2-YEAR LIMITED WARRANTY

Homedics sells its products with the intent that they are free of defects in manufacture and workmanship for a period of 2 years from the date of original purchase, except as noted below. Homedics warrants that its products will be free of defects in material and workmanship under normal use and service. This warranty extends only to consumers and does not extend to Retailers.

To obtain warranty service on your Homedics product, contact a Consumer Relations representative for assistance. Please make sure to have the model number of the product available.

Homedics does not authorize anyone, including but not limited to Retailers, the subsequent consumer purchaser of the product from a Retailer or remote purchasers, to obligate Homedics in any way beyond the terms set forth herein. This warranty does not cover damage caused by misuse or abuse; accident; the attachment of any unauthorized accessory; alteration to the product; improper installation; unauthorized repairs or modifications; improper use of electrical/power supply; loss of power; dropped product; malfunction or damage of an operating part from failure to provide manufacturer's recommended maintenance; transportation damage; theft; neglect; vandalism; or environmental conditions; loss of use during the period the product is at a repair facility or otherwise awaiting parts or repair; or any other conditions whatsoever that are beyond the control of Homedics.

This warranty is effective only if the product is purchased and operated in the country in which the product is purchased. A product that requires modifications or adoption to enable it to operate in any other country than the country for which it was designed, manufactured, approved, and/or authorized, or repair of products damaged by these modifications is not covered under this warranty.

THE WARRANTY PROVIDED HEREIN SHALL BE THE SOLE AND EXCLUSIVE WARRANTY. THERE SHALL BE NO OTHER WARRANTIES EXPRESS OR IMPLIED, INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS OR ANY OTHER OBLIGATION ON THE PART OF THE COMPANY WITH RESPECT TO PRODUCTS COVERED BY THIS WARRANTY. Homedics SHALL HAVE NO LIABILITY FOR ANY INCIDENTAL, CONSEQUENTIAL, OR SPECIAL DAMAGES. IN NO EVENT SHALL THIS WARRANTY REQUIRE MORE THAN THE REPAIR OR REPLACEMENT OF ANY PART OR PARTS THAT ARE FOUND TO BE DEFECTIVE WITHIN THE EFFECTIVE PERIOD OF THE WARRANTY. NO REFUNDS WILL BE GIVEN. IF REPLACEMENT PARTS FOR DEFECTIVE MATERIALS ARE NOT AVAILABLE, Homedics RESERVES THE RIGHT TO MAKE PRODUCT SUBSTITUTIONS IN LIEU OF REPAIR OR REPLACEMENT.

This warranty does not extend to the purchase of opened, used, repaired, repackaged, and/or resealed products, including but not limited to sale of such products on internet auction sites and/or sales of such products by surplus or bulk resellers. Any and all warranties or guarantees shall immediately cease and terminate as to any products or parts thereof that are repaired, replaced, altered, or modified, without the prior express and written consent of Homedics. This warranty provides you with specific legal rights. You may have additional rights that may vary from country to country. Because of individual country regulations, some of the above limitations and exclusions may not apply to you. For more information regarding our product line in the USA, please visit www.Homedics.com. For Canada, please visit www.Homedics.ca.

For Service in the USA

cservice@Homedics.com

8:30am-7:00pm EST Monday-Friday

1-800-466-3342

For Service in Canada

cservice@Homedicsgroup.ca

8:30am-5:00pm EST Monday-Friday

1-888-225-7378

©2022 Homedics, LLC. All rights reserved.

Homedics is a registered trademark of Homedics, LLC.

Distributed by Homedics, LLC, 3000 N Pontiac Trail, Commerce Township, MI 48390

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©2022 Homedics, LLC. Todos los derechos reservados.
Homedics son marcas registradas de Homedics, LLC.
Distribuido por Homedics, LLC, 3000 N Pontiac Trail, Commerce Township, MI 48390