# Homedics™ icyglide™ massage roller ball

## READ ALL INSTRUCTIONS BEFORE USING

- Close supervision is necessary when used by, on, or near children, invalids, or disabled persons.
- Use this appliance only for its intended use as described in this manual.
- Not for use by children. THIS IS NOT A TOY.
- NEVER operate this appliance if it has been damaged or if it is not working properly. Return it to the Homedics Service Center for examination and repair.
- Keep away from heated surfaces.

## SAVE THESE INSTRUCTIONS

### CAUTION: PLEASE READ ALL INSTRUCTIONS CAREFULLY BEFORE OPERATING.

- If you have concerns regarding your health, consult your doctor before using this product.
- Those with the following conditions should consult their doctor before using this product:
  - Pregnancy
  - Malignant tumor
  - Diabetes
  - Conditions requiring rest
  - Heart disease
  - Pacemakers
- If, at any time, soreness, rash, or pain occurs, discontinue use and consult your physician.
- NEVER use this product directly on swollen or inflamed areas, skin eruptions, or open wounds.
- A massage session longer than 15 minutes is not recommended.
- This product should NEVER be used by any individual suffering from any physical ailment that would limit the user's capacity to operate the controls or who has sensory deficiencies.

## INSTRUCTIONS FOR USE

1. This product can be used at either room temperature or cold. For cold therapy, simply place the entire product, or just the removable ball, in a freezer for at least 4 hours. After the ball is removed from the freezer, it will stay cold for up to 3 hours. **WARNING: Do not freeze the ball below a temperature of 0 degrees Fahrenheit. Doing so could result in damage to the product, or burning of the skin.**
2. To massage, simply grab the handle of the product and glide the ball across the skin. For best results, keep the product moving in a linear or circular motion across your body.
3. Apply pressure to the ball to work out tight knots on the body. Only apply as much pressure as you are comfortable with.
4. Lotion or oils can be used with the product, but ensure that the product is thoroughly cleaned after use.

## TO CLEAN

Clean using a damp, soft fabric cloth.

- NEVER use abrasive cleaners, brushes, gasoline, kerosene, glass/furniture polish, or paint thinner to clean. The ball can be removed from the handle for cleaning.

### CAUTION: All servicing of this massager must be performed by authorized Homedics Service Personnel only.

**USA** Email: [cservice@homedics.com](mailto:cservice@homedics.com)  
Phone: 1-800-466-3342 (8:30am-7:00pm EST Monday-Friday)

**CANADA** Email: [cservice@homedicsgroup.ca](mailto:cservice@homedicsgroup.ca)  
Phone: 1-888-225-7378 (8:30am-5:00pm EST Monday-Friday)

## LIMITED 1-YEAR WARRANTY

Homedics sells its products with the intent that they are free of defects in manufacture and workmanship for a period of 1 year from the date of original purchase, except as noted below. Homedics warrants that its products will be free of defects in material and workmanship under normal use and service. This warranty extends only to consumers and does not extend to Retailers.

To obtain warranty service on your Homedics product, contact a Consumer Relations representative for assistance. Please make sure to have the model number of the product available.

Homedics does not authorize anyone, including, but not limited to, Retailers, the subsequent consumer purchaser of the product from a Retailer or remote purchasers, to obligate Homedics in any way beyond the terms set forth herein. This warranty does not cover damage caused by misuse or abuse; accident; the attachment of any unauthorized accessory; alteration to the product; improper installation; unauthorized repairs or modifications; improper use of electrical/power supply; loss of power; dropped product; malfunction or damage of an operating part from failure to provide manufacturer's recommended maintenance; transportation damage; theft; neglect; vandalism; or environmental conditions; loss of use during the period the product is at a repair facility or otherwise awaiting parts or repair; or any other conditions whatsoever that are beyond the control of Homedics.

This warranty is effective only if the product is purchased and operated in the country in which the product is purchased. A product that requires modifications or adoption to enable it to operate in any other country than the country for which it was designed, manufactured, approved and/or authorized, or repair of products damaged by these modifications is not covered under this warranty.

THE WARRANTY PROVIDED HEREIN SHALL BE THE SOLE AND EXCLUSIVE WARRANTY. THERE SHALL BE NO OTHER WARRANTIES EXPRESS OR IMPLIED INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS OR ANY OTHER OBLIGATION ON THE PART OF THE COMPANY WITH RESPECT TO PRODUCTS COVERED BY THIS WARRANTY. HOMEDICS SHALL HAVE NO LIABILITY FOR ANY INCIDENTAL, CONSEQUENTIAL OR SPECIAL DAMAGES. IN NO EVENT SHALL THIS WARRANTY REQUIRE MORE THAN THE REPAIR OR REPLACEMENT OF ANY PART OR PARTS WHICH ARE FOUND TO BE DEFECTIVE WITHIN THE EFFECTIVE PERIOD OF THE WARRANTY. NO REFUNDS WILL BE GIVEN. IF REPLACEMENT PARTS FOR DEFECTIVE MATERIALS ARE NOT AVAILABLE, HOMEDICS RESERVES THE RIGHT TO MAKE PRODUCT SUBSTITUTIONS IN LIEU OF REPAIR OR REPLACEMENT.

This warranty does not extend to the purchase of opened, used, repaired, repackaged and/or resealed products, including but not limited to sale of such products on internet auction sites and/or sales of such products by surplus or bulk resellers. Any and all warranties or guarantees shall immediately cease and terminate as to any products or parts thereof which are repaired, replaced, altered, or modified, without the prior express and written consent of Homedics.

This warranty provides you with specific legal rights. You may have additional rights which may vary from state to state and country to country. Because of individual state and country regulations, some of the above limitations and exclusions may not apply to you.

For more information regarding our product line in the USA, please visit: [www.Homedics.com](http://www.Homedics.com). For Canada, please visit: [www.Homedics.ca](http://www.Homedics.ca).

©2019-2023 Homedics, LLC. All rights reserved.

Homedics is a registered trademark of Homedics, LLC.  
Distributed by Homedics, LLC, 3000 N Pontiac Trail, Commerce Township, MI 48390

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©2017-2023 Homedics, LLC. Todos los derechos reservados.
Homedics es una marca registrada de Homedics, LLC.
Distribuido por Homedics, LLC, 3000 N Pontiac Trail, Commerce Township, MI 48390