
**activate  
your product  
warranty**

scan the QR code and  
register your product  
within 7 days to activate  
your warranty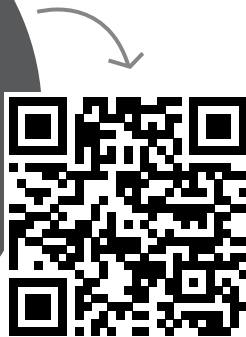

Registering also gives you access  
to tips, exciting offers and more!

portable back massage cushion  
with heat

**INSTRUCTION MANUAL AND WARRANTY INFORMATION**

VC-113H | L-04915, Rev. 1

# IMPORTANT SAFEGUARDS

WHEN USING ELECTRICAL PRODUCTS, ESPECIALLY WHEN CHILDREN ARE PRESENT, BASIC SAFETY PRECAUTIONS SHOULD ALWAYS BE FOLLOWED, INCLUDING THE FOLLOWING:

READ ALL INSTRUCTIONS BEFORE USING.

## **DANGER** – TO REDUCE THE RISK OF ELECTRIC SHOCK:

- Always unplug this appliance from the electrical outlet immediately after using and before cleaning.
- NEVER use pins or other metallic fasteners with this appliance.

## **WARNING** – TO REDUCE THE RISK OF BURNS, FIRE, ELECTRIC SHOCK, OR INJURY TO PERSONS:

- An appliance should never be left unattended when plugged in. Unplug from outlet when not in use, and before putting on or taking off parts.
- DO NOT operate under blanket or pillow. Excessive heating can occur and cause fire, electric shock, or injury to persons.
- Close supervision is necessary when this appliance is used by, on, or near children, invalids, or disabled persons.
- Use this appliance only for its intended use as described in this manual. DO NOT use attachments not recommended by Homedics; specifically, any attachments not provided with the unit.
- NEVER operate this appliance if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water. Return the appliance to a Homedics service center for examination and repair.
- DO NOT carry this appliance by supply cord or use cord as a handle.
- Keep the cord away from heated surfaces.
- NEVER operate the appliance with the air openings blocked. Keep the air openings free of lint, hair, and the like.
- NEVER drop or insert any object into any opening.
- DO NOT use outdoors.
- DO NOT operate where aerosol (spray) products are being used or where oxygen is being administered.
- To disconnect, turn all controls to the off position, then remove plug from outlet.

- Use heated surfaces carefully. May cause serious burns. DO NOT use over insensitive skin areas or in the presence of poor circulation. The unattended use of heat by children or incapacitated persons may be dangerous.
- All servicing of this product must be performed by authorized Homedics service personnel only.

## SAVE THESE INSTRUCTIONS

### **CAUTION** – PLEASE READ ALL INSTRUCTIONS CAREFULLY BEFORE OPERATING.

- This product is not intended for medical use. It is intended only to provide a luxurious massage.
- Consult your doctor prior to using this product if:
  - You are pregnant
  - You have a pacemaker
  - You have any concerns regarding your health
- NOT recommended for use by diabetics.
- NEVER leave the appliance unattended, especially if children are present.
- DO NOT use this product for more than 15 minutes at a time.
- Extensive use could lead to the product's excessive heating and shorter life. Should this occur, discontinue use and allow the unit to cool before operating.
- NEVER use this product directly on swollen or inflamed areas or skin eruptions.
- DO NOT use this product as a substitute for medical attention.
- NEVER use this product while in bed.
- This product should never be used by any individual suffering from any physical ailment that would limit the user's capacity to operate the controls or who has sensory deficiencies in the lower half of their body.
- This appliance is intended for household use only.

**NOTE:** For your safety, this product is equipped with a pressure-activated switch to prevent the heat function from staying on when the cushion is not in use. Heat will only operate if pressure is applied to this switch, located on the seat cushion of the unit (see page 5). Once pressure is removed, heat will turn off. After applying pressure, please allow 5–10 minutes for heat to reach maximum temperature.

**NOTE:** Not recommended for use on wooden surfaces as the zipper may damage the wood. Caution is also recommended when using on upholstered furniture.

## INSTRUCTIONS FOR USE

1. Plug adapter into a 120-volt AC outlet and plug the receptacle end into the input jack, located in the bottom of the integrated control on the cushion.
2. Attach the massage cushion to any chair using the integrated strapping system to secure the cushion.
3. Press the “MASSAGE” button on the control panel for low-intensity massage (Fig. 1). Press it again for high-intensity massage.
4. To activate the soothing heat function, press the “HEAT” button once (Fig. 1). Press the button again to deactivate the heat function.

NOTE: When selecting the heat feature, there will be a slight decrease in the strength of the massage output. This adjustment is perfectly normal and should not be interpreted as a defect.

5. To turn the unit off, press the massage button a third time.

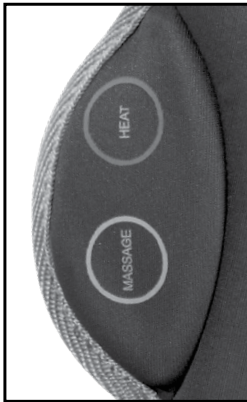
Fig. 1

### FOR USE IN YOUR CAR

1. Plug the 12-volt auto adapter into a 12V DC outlet, and plug the receptacle end into the input jack.
2. Slip the massage cushion straps over your seat.
3. Follow steps 3–5 above for operation.

## MAINTENANCE

### TO STORE

Place massager in its box or in a cool, dry place. Avoid contact with sharp edges or pointed objects that might cut or puncture the fabric surface. To avoid breakage, do not wrap the power cord around the unit. Do not hang the unit by the controller cord.

### TO CLEAN

Unplug the unit and allow it to cool before cleaning. Clean only with a soft, slightly damp sponge. Never allow water or any other liquids to come into contact with the unit.

- DO NOT immerse in any liquid to clean.
- NEVER use abrasive cleaners, brushes, gasoline, kerosene, glass/furniture polish, or paint thinner to clean.

**4 MASSAGING  
MOTORS**


Provide relaxing or  
invigorating massage.

**SOOTHING  
HEAT**

**PRESSURE-  
ACTIVATED  
SWITCH**

Apply pressure, then  
press the heat button  
in order to activate  
heat function.

**CHAIR  
STRAP  
SYSTEM**  
Easily  
fits most  
chairs.

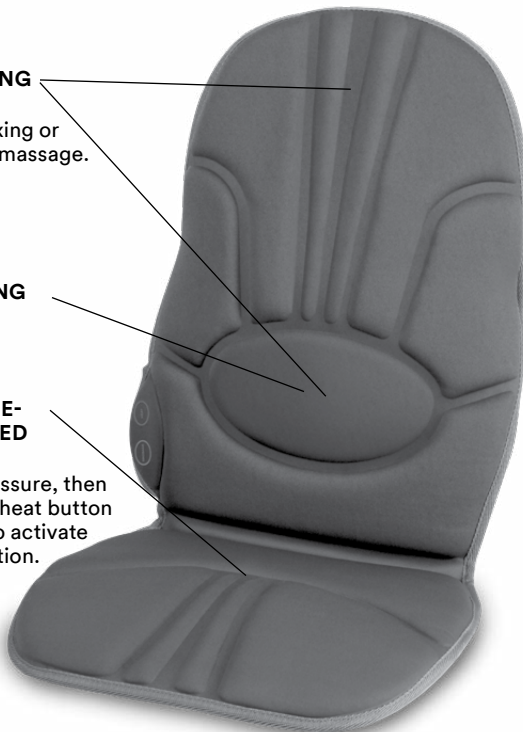# SUPPLIER'S DECLARATION OF CONFORMITY

**Product Description:** Portable Back Massage Cushion with Heat

**Model Number:** VC-113H

**Trade Name:** Homedics

## FCC COMPLIANCE STATEMENT

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) This device must accept any interference received, including interference that may cause undesired operation.

## U.S. CONTACT INFORMATION

**Company:** Homedics, LLC.

**Address:** 3000 N Pontiac Trail, Commerce Township, MI 48390

**8:30am-7:00pm EST Monday-Friday 1-800-466-3342**

Homedics is not responsible for any radio or TV interference caused by unauthorized modifications to this equipment. Such modifications could void the user authority to operate the equipment.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment to an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

**CAN ICES-003(B)/NMB-003(B)**

## 2-YEAR LIMITED WARRANTY

Homedics sells its products with the intent that they are free of defects in manufacture and workmanship for a period of 2 years from the date of original purchase, except as noted below. Homedics warrants that its products will be free of defects in material and workmanship under normal use and service. This warranty extends only to consumers and does not extend to Retailers.

To obtain warranty service on your Homedics product, contact a Consumer Relations representative for assistance. Please make sure to have the model number of the product available.

Homedics does not authorize anyone, including but not limited to Retailers, the subsequent consumer purchaser of the product from a Retailer or remote purchasers, to obligate Homedics in any way beyond the terms set forth herein. This warranty does not cover damage caused by misuse or abuse; accident; the attachment of any unauthorized accessory; alteration to the product; improper installation; unauthorized repairs or modifications; improper use of electrical/power supply; loss of power; dropped product; malfunction or damage of an operating part from failure to provide manufacturer's recommended maintenance; transportation damage; theft; neglect; vandalism; or environmental conditions; loss of use during the period the product is at a repair facility or otherwise awaiting parts or repair; or any other conditions whatsoever that are beyond the control of Homedics.

This warranty is effective only if the product is purchased and operated in the country in which the product is purchased. A product that requires modifications or adoption to enable it to operate in any other country than the country for which it was designed, manufactured, approved, and/or authorized, or repair of products damaged by these modifications is not covered under this warranty.

THE WARRANTY PROVIDED HEREIN SHALL BE THE SOLE AND EXCLUSIVE WARRANTY. THERE SHALL BE NO OTHER WARRANTIES EXPRESS OR IMPLIED, INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS OR ANY OTHER OBLIGATION ON THE PART OF THE COMPANY WITH RESPECT TO PRODUCTS COVERED BY THIS WARRANTY. HOMEDICS SHALL HAVE NO LIABILITY FOR ANY INCIDENTAL, CONSEQUENTIAL, OR SPECIAL DAMAGES. IN NO EVENT SHALL THIS WARRANTY REQUIRE MORE THAN THE REPAIR OR REPLACEMENT OF ANY PART OR PARTS THAT ARE FOUND TO BE DEFECTIVE WITHIN THE EFFECTIVE PERIOD OF THE WARRANTY. NO REFUNDS WILL BE GIVEN. IF REPLACEMENT PARTS FOR DEFECTIVE MATERIALS ARE NOT AVAILABLE, HOMEDICS RESERVES THE RIGHT TO MAKE PRODUCT SUBSTITUTIONS IN LIEU OF REPAIR OR REPLACEMENT.

This warranty does not extend to the purchase of opened, used, repaired, repackaged, and/or resealed products, including but not limited to sale of such products on internet auction sites and/or sales of such products by surplus or bulk resellers. Any and all warranties or guarantees shall immediately cease and terminate as to any products or parts thereof that are repaired, replaced, altered, or modified, without the prior express and written consent of Homedics.

This warranty provides you with specific legal rights. You may have additional rights that may vary from country to country. Because of individual country regulations, some of the above limitations and exclusions may not apply to you.

For more information regarding our product line in the USA, please visit [www.Homedics.com](http://www.Homedics.com). For Canada, please visit [www.Homedics.ca](http://www.Homedics.ca).

---

### FOR SERVICE IN THE USA

[cservice@Homedics.com](mailto:cservice@Homedics.com)

8:30am-7:00pm EST Monday-Friday

1-800-466-3342

### FOR SERVICE IN CANADA

[cservice@Homedics.ca](mailto:cservice@Homedics.ca)

8:30am-5:00pm EST Monday-Friday

1-888-225-7378

©2024 Homedics, LLC. All rights reserved.

Homedics is a registered trademark of Homedics, LLC.

Distributed by Homedics, LLC, 3000 N Pontiac Trail, Commerce Township, MI 48390

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©2024 Homedics, LLC. Todos los derechos reservados.
Homedics es una marca registrada de Homedics, LLC.
Distribuido por Homedics, LLC, 3000 N Pontiac Trail, Commerce Township, MI 48390