

Face Mask – Single Use



Model #: MSK-F1CO

FEATURES


- ≥95 PFE
- Helps protect against odor and dust
- 3-layer protection
 - Outer Layer: Non-woven fabric
 - Middle Layer: Melt-blown fabric
 - Inner Layer: Soft for your skin and absorbs the mist you breathe out
- Eco-friendly and adjustable plastic clip for a proper fit
- Disposable and maintenance-free
- Color: blue
- Main Material: non-woven fabric + melt-blown fabric + non-woven fabric
- Latex Free

APPROVALS AND STANDARDS

- FDA, product code QKR
- Health Canada, Class I, PPE

MATERIAL COMPOSITION

- Material: polypropylene

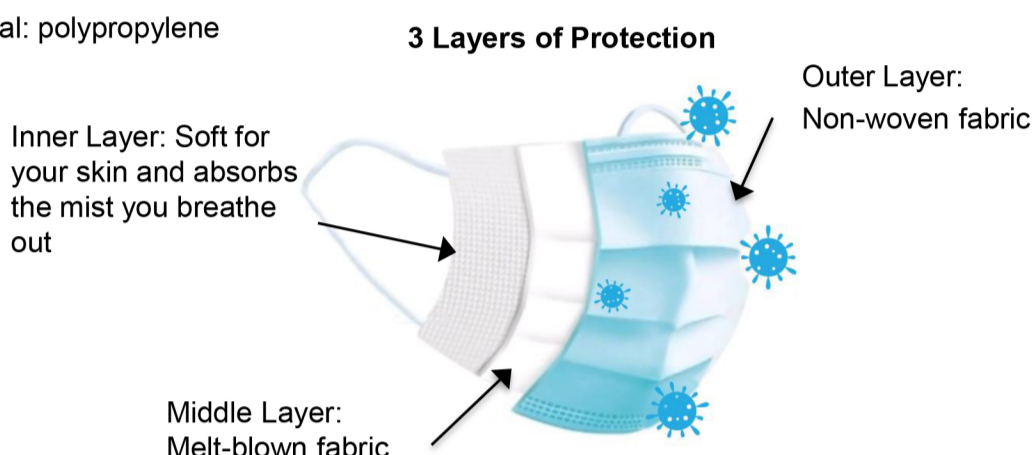

PRODUCT USE INFORMATION

HOW TO USE:

- 1) Remove mask from bag. Unfold mask and apply to face from bridge of nose down to jaw.
- 2) Lightly press along nose piece to fit.
- 3) Hang elastic straps over both ears.
- 4) Adjust to fit mask to cover from bridge of nose to jaw.
- 5) Dispose after use or if it becomes soiled. Single use only. Do not reuse.

MODE D'EMPLOI

- 1) Retirez le masque du sachet. Dépliez le masque et placez-le sur le visage en couvrant la zone de l'arête du nez à la mâchoire.
 - 2) Appuyez légèrement sur la partie couvrant le nez pour assurer un bon ajustement.
 - 3) Accrochez les brides élastiques autour des oreilles.
 - 4) Ajustez le masque pour qu'il couvre la zone de l'arête du nez à la mâchoire.
 - 5) Jetez le masque après l'utilisation ou s'il est sale.
- Réservé à un usage unique. Ne pas réutiliser.



CAUTION

- Caution: This product is not intended for use except as indicated. Please refer to www.HoMedics.com/learn/face-masks for additional information.

MISE EN GARDE:

Ce produit n'est pas destiné à être utilisé sauf indication contraire. Veuillez consulter www.HoMedics.com/learn/face-masks pour plus d'informations.

MASK PRODUCT SPECIFICATIONS

Product Dims	6.89" x 3.74"
Product Weight	.0006 lbs
Shelf Life	5 years from manufacture

MSK-F1CO-10B PRODUCT SPECIFICATIONS

Package Dims	7.13" x 3.90" x 0.87"
Case Dims.	8.11" x 5.53" x 7.75" EST
Cube Dims.	0.20 ft ³
Package Weight	0.19 lbs. EST
Case Weight	2.53 lbs. EST
Units per Pouch	10
Boxes in Master	12
UPC	0 31262 09972 3

MSK-F1CO-20B PRODUCT SPECIFICATIONS

Package Dims	7.13" x 3.90" x 1.38"
Case Dims.	8.58" x 7.68" x 9.53" EST
Cube Dims.	0.31 ft ³
Package Weight	0.26 lbs EST
Case Weight	4.02 lbs. EST
Units per Box	20
Boxes in Master	12
UPC	0 31262 09973 0

MSK-F1CO-50B PRODUCT SPECIFICATIONS

Package Dims	7.13" x 3.90" x 3.11"
Case Dims.	14.88" x 8.58" x 10.39" EST
Cube Dims.	0.69 ft ³
Package Weight	0.49 lbs. EST
Case Weight	7.49 lbs. EST
Units per Box	50
Boxes in Master	12
UPC	0 31262 09974 7

