HOMEDICS®
THERA-P®

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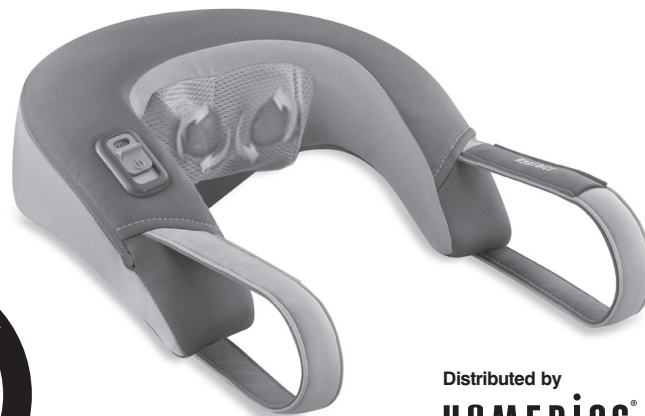
IB-NMS230THP

HOMEDICS®
THERA-P®

Neck & Shoulder Shiatsu Massager WITH HEAT

Distributed by

HOMEDICS®

HoMedics, LLC.
3000 Pontiac Trail
Commerce Township, MI
48390

**Instruction Manual and
Warranty Information**

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2 year
limited warranty

IMPORTANT SAFETY INSTRUCTIONS

WHEN USING ELECTRICAL PRODUCTS, ESPECIALLY WHEN CHILDREN ARE PRESENT, BASIC SAFETY PRECAUTIONS SHOULD ALWAYS BE FOLLOWED, INCLUDING THE FOLLOWING:

READ ALL INSTRUCTIONS BEFORE USING

DANGER – TO REDUCE THE RISK OF ELECTRIC SHOCK:

- Always unplug the appliance from the electrical outlet immediately after using and before cleaning.
- DO NOT reach for an appliance that has fallen into water. Unplug it immediately.
- DO NOT use while bathing or in the shower.
- DO NOT place or store appliance where it can fall or be pulled into a tub or sink.
- DO NOT place into water or other liquid.

WARNING – TO REDUCE THE RISK OF BURNS, ELECTRIC SHOCK, FIRE OR INJURY TO PERSONS:

- An appliance should NEVER be left unattended when plugged in. Unplug from outlet when not in use and before putting on or taking off parts or attachments.
- NOT for use by children.
- Close supervision is necessary when this appliance is used by, on or near invalids or disabled persons.
- Use this appliance only for its intended use as described in this manual. DO NOT use attachments not recommended by HoMedics; specifically any attachments not provided with the unit.
- NEVER operate this appliance if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water. Return it to HoMedics Service Center for examination and repair.
- Keep cord away from heated surfaces.

Caution:
All servicing of this massager must be performed by authorized HoMedics service personnel only.

- NEVER drop or insert any object into any opening.
- DO NOT operate where aerosol (spray) products are being used or where oxygen is being administered.
- DO NOT operate under a blanket or pillow. Excessive heating can occur and cause fire, electric shock or injury to persons.
- DO NOT carry this appliance by the power cord or use cord as handle.
- To disconnect, turn all controls to the "off" position, then remove plug from outlet.
- DO NOT use outdoors.
- NEVER operate the appliance with the air openings blocked. Keep the air openings free of lint, hair, and the like.
- NEVER operate on a soft surface such as a bed or couch where the air openings may be blocked.
- Use heated surfaces carefully. May cause serious burns. DO NOT use over insensitive skin areas or in the presence of poor circulation. The use of heat by children or incapacitated persons may be dangerous.
- DO NOT use this massager in close proximity to loose clothing, jewelry, or long hair.

SAVE THESE INSTRUCTIONS

CAUTION—PLEASE READ ALL INSTRUCTIONS CAREFULLY BEFORE OPERATING.

- This product is not intended for medical use. It is intended only to provide a luxurious massage.
- NOT for use by children.
- Consult your doctor prior to using this product, if
 - You are pregnant
 - You have a pacemaker
 - You have any concerns regarding your health
- NOT recommended for use by Diabetics.
- NEVER leave the appliance unattended, especially if children are present.
- NEVER cover the appliance when it is in operation.
- DO NOT use this product for more than 15 minutes at a time.
- Extensive use could lead to the product's excessive heating

Do not jam or force any part of your body in the moving massage mechanism.

- and shorter life. Should this occur, discontinue use and allow the unit to cool before operating.
- NEVER use this product directly on swollen or inflamed areas or skin eruptions.
 - DO NOT use this product as a substitute for medical attention.
 - The product is equipped with overheating protection circuit, if the power is cut-off, please wait until the product is cooled down.
 - DO NOT use this product before bed. The massage has a stimulating effect and can delay sleep.
 - NEVER use this product while in bed.
 - This product should NEVER be used by any individual suffering from any physical ailment that would limit the user's capacity to operate the controls.
 - NEVER use this product in automobiles.
 - This appliance is intended for household use only.

Instructions for Use

Fig. 1

To reduce the risk of electric shock, this appliance is equipped with a polarized plug (one blade wider than the other). This plug will fit in a polarized outlet only one way. If the plug does not fit fully in the outlet, reverse the plug. If it still does not fit, contact a qualified electrician to install the proper outlet. Do not change the plug in any way.

1. Insert the power cord into a 120 volt AC outlet. To activate the Shiatsu massage action, slide switch up one notch. To activate soothing heat with the massage, slide switch up one more notch (Fig. 1).

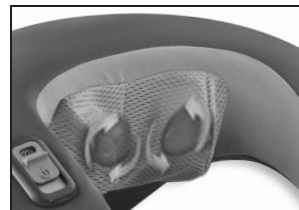
Fig. 2

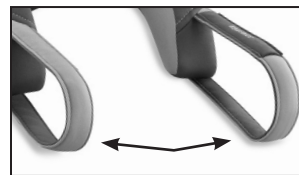
Fig. 3

2. When activated to Shiatsu & Heat, the massage heads will glow red (Fig. 2). Allow some time for the heating action to warm up. When finished slide switch back to bottom position.

NOTE: For more intense massage place hands in flex handles and gently pull forward (Fig 3).

Maintenance

To Store

Place massager in its box or in a safe, dry, cool place. Avoid contact with sharp edges or pointed objects which might cut or puncture the fabric surface. To avoid breakage, do not wrap the power cord around the unit. Do not hang the unit by the power cord or flex handles.

To Clean

Unplug the unit and allow it to cool before cleaning. Spot clean only with a soft, slightly damp cloth. Never allow water or any other liquids to come into contact with the unit.

- Do not immerse in any liquid to clean.
- Never use abrasive cleaners, brushes, gasoline, kerosene, glass/furniture polish or paint thinner to clean.
- Do not attempt to repair the Shiatsu Neck Massager. There are no user serviceable parts. For service, call the Consumer Relations telephone number listed in the Warranty Section.

Neck & Shoulder Shiatsu Massager WITH HEAT

Power Switch

To turn the massager on, slide switch up one notch for Shiatsu Massage. For massage with heat, slide switch up one more notch. To turn the unit off, slide switch back to bottom position.

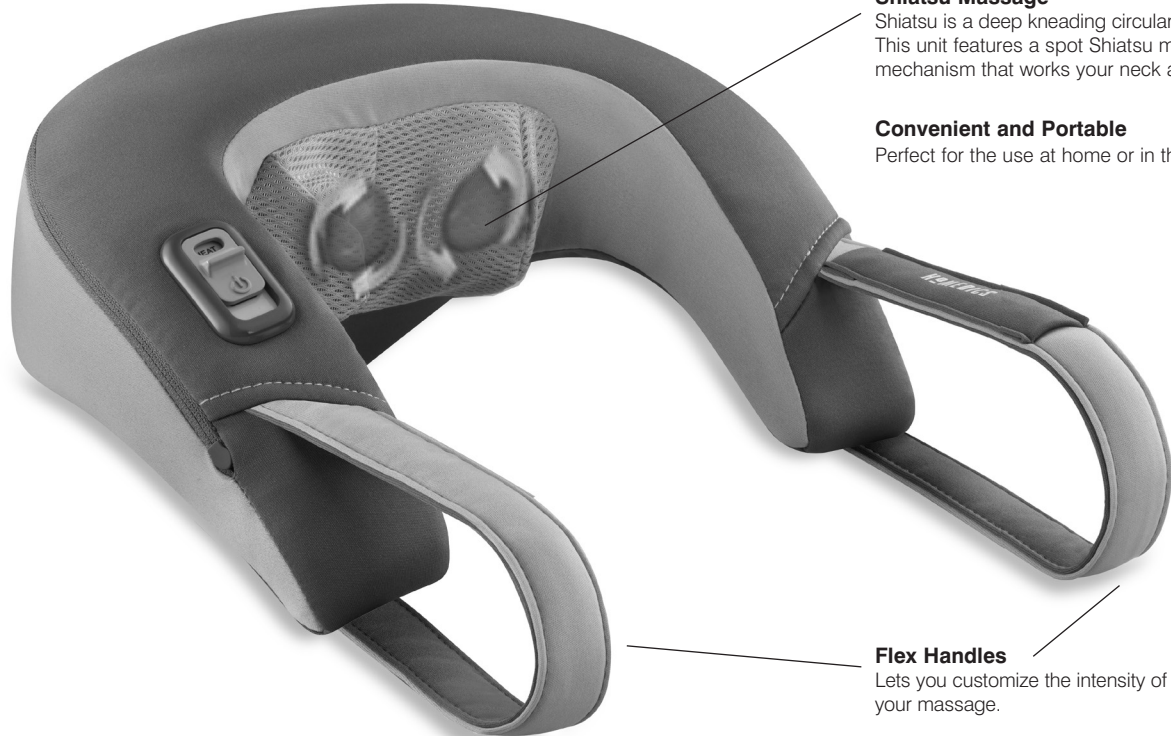
Soothing Heat

For a more relaxing massage.

Power Supply Cord

The Neck & Shoulder Shiatsu Massager is powered by a 120V power cord.

- Do not attempt to repair the Neck & Shoulder Shiatsu Massager. There are no user serviceable parts. For service, call the Consumer Relations telephone number listed in the Warranty Section.

Shiatsu Massage

Shiatsu is a deep kneading circular massage. This unit features a spot Shiatsu massage mechanism that works your neck area.

Convenient and Portable

Perfect for the use at home or in the office.

Flex Handles

Lets you customize the intensity of your massage.

HOMEDICS®
THERA-P®

Email:
cservice@homedics.com

Monday – Friday
8:30am – 7:00pm (EST)

1.800.466.3342

TWO YEAR LIMITED WARRANTY
(Valid in USA only)

HoMedics sells its products with the intent that they are free of defects in manufacture and workmanship for a period of two years from the date of original purchase, except as noted below. HoMedics warrants that its products will be free of defects in material and workmanship under normal use and service. This warranty extends only to consumers and does not extend to Retailers.

To obtain warranty service on your HoMedics product, contact a consumer relations representative by telephone at 1-800-466-3342 for assistance. Please make sure to have the model number of the product available.

HoMedics does not authorize anyone, including, but not limited to, Retailers, the subsequent consumer purchaser of the product from a Retailer or remote purchasers, to obligate HoMedics in any way beyond the terms set forth herein. This warranty does not cover damage caused by misuse or abuse; accident, the attachment of any unauthorized accessory; alteration to the product; improper installation, unauthorized repairs or modifications; improper use of electrical/power supply; loss of power; dropped product; malfunction or damage of an operating part from failure to provide manufacturer's recommended maintenance; transportation damage; theft; neglect, vandalism; or environmental conditions; loss of use during the period the product is at a repair facility or otherwise awaiting parts or repair; or any other conditions whatsoever that are beyond the control of HoMedics.

This warranty is effective only if the product is purchased and operated in the country in which the product is purchased. A product that requires modifications or adoption to enable it to operate in any other country than the country for which it was designed, manufactured, approved and/or authorized, or repair of products damaged by these modifications is not covered under this warranty.

THE WARRANTY PROVIDED HEREIN SHALL BE THE SOLE AND EXCLUSIVE WARRANTY. THERE SHALL BE NO OTHER WARRANTIES EXPRESS OR IMPLIED INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS OR ANY OTHER OBLIGATION ON THE PART OF THE COMPANY WITH RESPECT TO PRODUCTS COVERED BY THIS WARRANTY. HOMEDICS SHALL HAVE NO LIABILITY FOR ANY INCIDENTAL, CONSEQUENTIAL OR SPECIAL DAMAGES. IN NO EVENT SHALL THIS WARRANTY REQUIRE MORE THAN THE REPAIR OR REPLACEMENT OF ANY PART OR PARTS WHICH ARE FOUND TO BE DEFECTIVE WITHIN THE EFFECTIVE PERIOD OF THE WARRANTY. NO REFUNDS WILL BE GIVEN. IF REPLACEMENT PARTS FOR DEFECTIVE MATERIALS ARE NOT AVAILABLE, HOMEDICS RESERVES THE RIGHT TO MAKE PRODUCT SUBSTITUTIONS IN LIEU OF REPAIR OR REPLACEMENT.

This warranty does not extend to the purchase of opened, used, repaired, repackaged and/or resealed products, including but not limited to sale of such products on Internet auction sites and/or sales of such products by surplus or bulk resellers. Any and all warranties or guarantees shall immediately cease and terminate as to any products or parts thereof which are repaired, replaced, altered, or modified, without the prior express and written consent of HoMedics.

This warranty provides you with specific legal rights. You may have additional rights which may vary from country to country. Because of individual country regulations, some of the above limitations and exclusions may not apply to you.

For more information regarding our product line in the USA, please visit: www.homedics.com

HoMedics and Thera P are registered trademarks of HoMedics, LLC.
©2013 HoMedics, LLC. All rights reserved.

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