activate your warranty and meet your product

Scan the QR code **within 7 days** of receiving your product to activate your warranty and experience all the benefits!

body composition glo smart scale
with ambient illumination

INSTRUCTION MANUAL AND WARRANTY INFORMATION

SCL-BBC200, SCL-BBC250, SCL-BBC350, SCL-XBC200 | L-05257, Rev. 2

IMPORTANT SAFETY INSTRUCTIONS:

WHEN USING ELECTRICAL PRODUCTS, ESPECIALLY WHEN CHILDREN ARE PRESENT, BASIC SAFETY PRECAUTIONS SHOULD ALWAYS BE FOLLOWED, INCLUDING THE FOLLOWING:

READ AND SAVE THESE INSTRUCTIONS.

- DO NOT use if you have implanted medical devices such as a cardiac pacemaker or artificial lung.
- DO NOT use while wearing devices such as a wearable electrocardiograph.
- DO NOT use if you are pregnant.
- Keep batteries away from children.
- DO NOT use on slippery floor.
- This product is intended for use as a body composition analyzer for personal fitness and health monitoring only.
- This product should not be used for treating or diagnosing any medical conditions.
- Always consult your physician or healthcare professional prior to making any changes to your diet or exercise regimen.
- DO NOT reach for a unit that has fallen into water.
- DO NOT use while bathing or while in a shower.
- DO NOT place or store appliance where it can fall or be pulled into a tub or sink.
- DO NOT place in or drop into water or other liquid.
- DO NOT use outdoors. This product is intended for household use only.
- DO NOT strike or drop the scale.
- DO NOT stand on the edge of the scale.
- DO NOT jump on the scale.
- DO NOT step onto a wet scale. Wet glass is slippery. Always ensure the scale surface is dry and feet are dry before use.
- Always place the scale on a hard, dry, flat surface before measurement.
- Check the scale before use. DO NOT use the scale if the glass is cracked or broken. If you see a crack, contact Customer Support.
- DO NOT overload the scale. Maximum weight capacity is 400lbs (180kg or 28st:7lb).
- Close supervision is necessary when this appliance is used by, on, or near children, invalids, or disabled persons.
- The measurement data may not be accurate if using the scale immediately after intense exercises.
- DO NOT modify this equipment without authorization of the manufacturer.
- This scale is not field serviceable. You should not use any tool to open the device nor should you attempt to adjust anything inside the device. If you have any problems with this device, please contact Homedics Consumer Relations (contact information on warranty page).

Operating Environment	Temperature: 59°F - 104°F (15°C - 40°C) Humidity: 30 - 80% RH Atmospheric Pressure: 970hPa - 1050hPa
Storage Environment	Temperature: -31°F - 140°F (-35°C - 60°C) Humidity: 10 - 90% RH Atmospheric Pressure: 700hPa - 1060hPa


SYMBOL	EXPLANATION
	This symbol means "Manufactured by" and provides the name and address of the legal manufacturer.
	Date of manufacture.
	This symbol refers to the batch code of the device.
	Type BF Applied Part, BF indicates that it is used for the body. This device is connected to the main voltage.
	This symbol indicates to refer to instruction manual/ booklet
	This symbol indicates the product may not be discarded as household waste.
	This symbol indicates the product is designed for indoor use only.

Upon opening the product package, you will find the following contents inside:


ITEMS	QUANTITY	UNIT
MAIN UNIT	1	PCS
USER MANUAL	1	PCS
QUICK START GUIDE	1	PCS
BATTERIES	4 (SCL-BBC200, SCL-BBC250 AND SCL-BBC350) 3 (SCL-XBC200)	PCS

GET TO KNOW YOUR GLO BODY COMPOSITION SCALE WITH AMBIENT ILLUMINATION

SCL-BBC200, SCL-BBC250 AND SCL-BBC350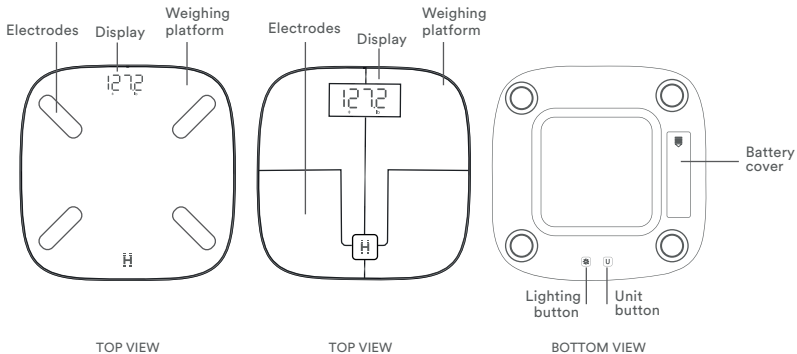

FOR SCL-XBC200

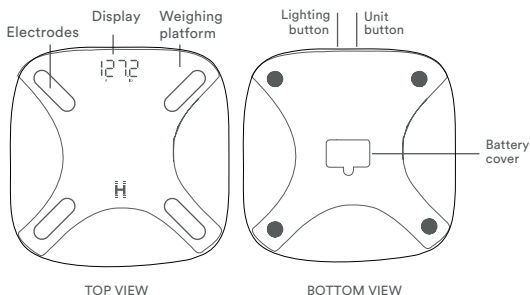BEFORE TURNING THE UNIT ON

This scale is shipped with 4 AAA high capacity alkaline (1100 mAh) batteries for the SCL-BBC200, SCL-BBC250 and SCL-BBC350 or 3 AAA high capacity alkaline (1100 mAh) batteries for the SCL-XBC200. Please install the batteries inside the battery compartment to power on your scale before use. After power on, you can take your first measurement.

BACK PANEL BUTTONS

LIGHTING BUTTON

Press the lighting button to choose your lighting brightness: high, low, or off
NOTE: Setting lighting to low will improve battery life

UNIT BUTTON


Press to choose your desired units of measure: lb, kg, or st:lb

DISPLAY SCREENS

Upon measurement, weight, body mass index (BMI) and body fat percentage (%) will display on the scale's display screen for 4 seconds each. Download the Homedics Scales App for all 13 essential biometrics.

NOTE: If you have not connected your scale to the Homedics Scales App prior to measurement and input your height, date of birth, and sex, the scale will only display weight. BMI and body fat % will only show on the scale screen after connecting to the Homedics Scales App and inputting your height, date of birth, and sex.


Weight

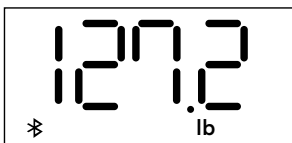Body Mass Index

Body Fat Percentage

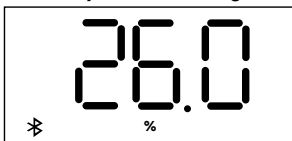HOW TO CONNECT THE MOBILE APP

1. Download the Homedics Scales app in the App Store® or Google Play store or scan the QR code.
2. Open the Homedics Scales app. If you already have an account, log in. To create a new account, tap Sign Up.
3. To add a new device, tap “+ Pair the device”.
4. Follow the in-app instructions to add your scale.

HOW TO TAKE A MEASUREMENT

NOTE: To use athlete mode or baby mode, follow the prompts in the app BEFORE taking your first measurement.

1. Open your Homedics Scales App. Then step on the scale.

2. Wait while the app syncs your measurement data.

3. Once the app shows all 13 measurements, you can step off the scale.

4. Tap your measurements to get more information.


AMBIENT ILLUMINATION

1. Install 4 AAA high capacity alkaline (1100 mAh) batteries for the SCL-BBC200, SCL-BBC250 and SCL-BBC350, or 3 AAA high capacity alkaline (1100 mAh) batteries for the SCL-XBC200 and place scale on a hard, flat surface.
2. Once you are within the proximity zone, scale's lighting will turn on for 15 seconds. If you stay in the proximity zone, the lights will stay on for an additional 30 seconds (total of 45 seconds) and then turn off for 10 minutes. Lighting will always turn on when taking a measurement.
3. If the LEDs don't turn on when you are close to the scale, click the lighting button located on the bottom of the scale to turn the night light on.
4. If you want to dim the motion sensing night light, click the lighting button once located on the bottom of the scale.
5. If you ever want to turn the motion sensing night light off, click the lighting button located on the bottom of the scale again.

NOTE: To improve battery life, use the low level brightness setting and keep the scale in a low traffic area.

HOW TO USE

1. Place scale on a hard, flat, dry surface. Avoid carpet or other soft surfaces.
CAUTION: Placing the scale on soft surfaces such as shag carpet may cause tipping or inaccurate results.
2. Step on the scale with both feet. Use bare feet for best results.
3. Stand on the scale until the display flashes several times. This is your final measurement.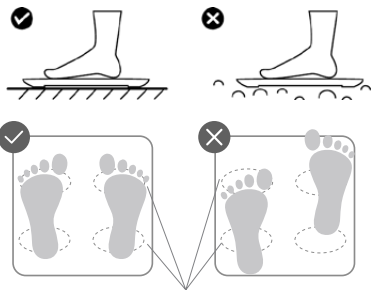

MEASUREMENT TIPS

This product employs BIA (Bio-electrical Impedance Analysis) technology. To ensure accurate measurement results, follow these instructions:

- Use the scale at the same time each day.
- Measure under the same conditions as much as possible.
- Measure with clean, bare feet. Wearing socks or having soiled feet will impact the accuracy of your results.

Avoid measuring under the following conditions:


- During or immediately after strenuous exercise
- Immediately after showering, bathing, or using a sauna
- After heavy alcohol consumption
- During or immediately after eating large meals
- While experiencing a fever.

CLEANING THE DIGITAL SCALE


- Use a slightly damp cloth or disinfecting wipe to clean the surface of the scale, followed by a soft, dry cloth to wipe away any remaining moisture.
- DO NOT use abrasive cleaning agents to clean the scale.

BATTERY REPLACEMENT


FOR SCL-BBC200, SCL-BBC250 AND SCL-BBC350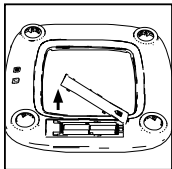


1. Remove the battery cover by sliding the cover in the direction of the open arrow and lifting the top edge.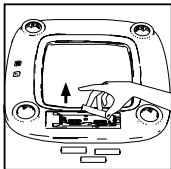

2. Remove the batteries and dispose of per local regulations.

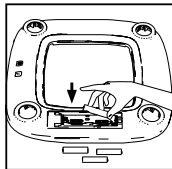
3. Install 4 new AAA high capacity alkaline (1100 mAh) batteries following the polarity as shown.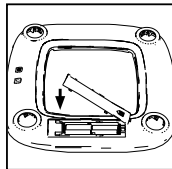


4. Replace the battery cover.


FOR SCL-XBC200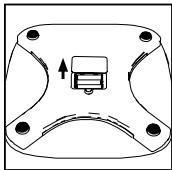


1. Remove the battery cover by pinching the tab and lifting the cover.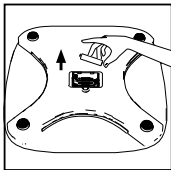

2. Remove the batteries and dispose of per local regulations.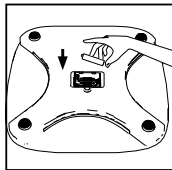

3. Install 3 new AAA high capacity alkaline (1100 mAh) batteries following the polarity as shown.

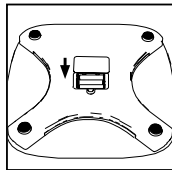4. Replace the battery cover.

BATTERY PRECAUTIONS

⚠ WARNING — KEEP BATTERIES OUT OF REACH OF CHILDREN.

1. Replace all batteries at the same time. Do not mix new and used batteries. Do not mix alkaline, standard, or rechargeable batteries. Use only the size and type of batteries specified.
2. Clean the battery contacts and the contacts of the device with a dry cloth prior to installing the batteries.
3. Ensure the batteries are installed correctly with regard to polarity (+/-).
4. Remove batteries from the device when the device is not used for an extended period.
5. Remove batteries promptly from the device when the batteries are depleted.
6. Keep batteries away from children.
7. Batteries are small objects and could be ingested. If swallowed, contact a physician at once.
8. Remove depleted batteries from the unit promptly. Dispose of used batteries safely.
9. Recycle or dispose of batteries properly in accordance with all local, state, province, and country regulations. Do not open batteries or dispose of in fire.

BIOMETRICS DESCRIPTIONS

WEIGHT

Body weight is a simple measure of total body components and is compared to a “standard” or desirable weight.

BODY FAT %

Body fat (BF) percentage refers to the ratio of fat (adipose tissue) in the body in comparison to overall body weight. While body mass index (BMI) is often used to estimate body fat, it’s not always an accurate method of doing so. According to the American Council on Exercise (ACE), a body fat range of 25% to 31% for women or 18% to 24% for men is typical for the average nonathlete. A BF below 14% for women or 6% for men may be considered dangerously low and can lead to health risks.

	Essential	Athletes	Fitness	Acceptable	High	Excess
MEN	2%-6%	6%-13%	13%-17%	17%-25%	25%-32%	≥32%
WOMEN	10%-14%	14%-21%	21%-25%	25%-32%	32%-38%	≥38%

BMI

A standardized ratio of weight to height, used as a general indicator of health.

The Body Mass Index (BMI) is a widely used health indicator. It can be roughly calculated by dividing your body weight (in kilograms) by your height (in meters) squared. If the resulting figure is less than 18.5, you are underweight. A number between 18.5 and 25 indicates a healthy weight. A number above 25 is overweight and a number above 30 is obese. Although BMI is a generally accepted health indicator, it is not the only one. For example, someone with a lot of muscle mass can have a high BMI without being unhealthy. By looking at the ratio of muscle to fat mass, among other things, you get a much better understanding of your health.

LOW	STANDARD	HIGH	EXCESS
< 18.5	18.5 - 25	25 - 30	≥ 30

Software Version:

SCL-BBC200,SCL-BBC250,SCL-BBC350—5.0.2.1, SCL-XBC200—7.0.2.1

Expected service life: 5 years

SPECIFICATIONS

Model	SCL-BBC200, SCL-BBC250, SCL-BBC350, SCL-XBC200
Weight Units	lb / kg / st:lb
Weight Capacity	400lb / 180kg / 28st 7lb
Increments	lb: 0.1lb kg: 0.05kg for less than 100kg total weight, 0.1kg for 100kg or greater total weight st:lb: 0.1lb for less than 20st, 1lb for 20st or greater
Platform	Tempered Glass
Weight Display Auto-Off	10 Seconds
Dimensions	SCL-BBC200: 11.8×11.8×1.1in / 30×30×2.9cm SCL-BBC250: 11.8×11.8×1.1in / 30×30×2.9cm SCL-BBC350: 11.8×11.8×1.1in / 30×30×2.9cm SCL-XBC200: 11.8×11.8×1.1in / 30×30×2.8cm

TROUBLESHOOTING

PROBLEM	POSSIBLE CAUSE	SOLUTION
Scale won't turn on or no display	<ul style="list-style-type: none"> Batteries are dead Batteries aren't fully installed Scale isn't on a hard, flat surface 	<ul style="list-style-type: none"> Use new batteries Remove battery cover to check if all batteries are installed in the correct direction Place your scale on a hard, flat surface
Lights won't turn on	<ul style="list-style-type: none"> Batteries are dead Batteries aren't fully installed Lights are turned off 	<ul style="list-style-type: none"> Replace batteries with new batteries Remove battery cover to check if all batteries are installed in the correct direction Click the light button located on the bottom of the scale
The scale readings are unstable or inaccurate	<ul style="list-style-type: none"> Scale isn't on a hard, flat surface 	<ul style="list-style-type: none"> Place your scale on a hard, flat surface
Display shows: 	<ul style="list-style-type: none"> Batteries are dead 	<ul style="list-style-type: none"> Replace batteries with new batteries To extend battery life: use high capacity alkaline batteries (1100 mAh), keep the scale in a low traffic area and keep the scales brightness on low
Display shows: 	<ul style="list-style-type: none"> Weight applied to the scale is greater than 400 lbs / 181 kg / 28st 7lb 	<ul style="list-style-type: none"> Step off the scale. DO NOT apply weight greater than 400lbs / 180kg / 28st 7lb to the scale

SUPPLIER'S DECLARATION OF CONFORMITY

Product Description: BODY COMPOSITION GLO SMART SCALE WITH AMBIENT ILLUMINATION

Model Number: SCL-BBC200, SCL-BBC250, SCL-BBC350, SCL-XBC200

Trade Name: Homedics

FCC Compliance Statement

This device complies with Part 15 of the FCC Rules. This device contains license-exempt transmitter/receiver that complies with Innovation, Science and Economic Development Canada's license-exempt RSS(s). Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) This device must accept any interference received, including interference that may cause undesired operation.

U.S. Contact Information

Company: Homedics, LLC.

Address: 3000 N Pontiac Trail, Commerce Township, MI 48390

8:30am-7:00pm EST Monday-Friday 1-800-466-3342

Homedics is not responsible for any radio or TV interference caused by unauthorized modifications to this equipment. Such modifications could void the user authority to operate the equipment.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment to an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

This product has been tested and complies with the requirements for Federal Communication Commission, Part 15. Although this product is tested and complies with FCC, it may interfere with other devices. If this product is found to interfere with another device, separate the other device and this product. Conduct only the user maintenance found in this instruction manual. Other maintenance and servicing may cause harmful interference and can void the required FCC compliance.

CAN ICES-003(B)/NMB-003(B)

RF Exposure Statement

This equipment complies with FCC and ISED RF radiation exposure limits set forth for an uncontrolled environment. This transmitter must not be co-located or operated in conjunction with any other antenna or transmitter.

ELECTROMAGNETIC COMPATIBILITY

WARNING: Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation

WARNING: Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the [scale] including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

Guidance and manufacture's declaration – electromagnetic emission			
The scale is intended for use in the electromagnetic environment specified below. The customer of the user of the scale should assure that it is used in such an environment.			
Emission test	Compliance	Electromagnetic environment – guidance	
Conducted and radiated rf emissions CISPR 11	Group 1	The scale uses rf energy only for its internal function. Therefore, its rf emissions are very low and are not likely to cause any interference in nearby electronic equipment.	
Conducted RF emissions CISPR 11	Not applicable	The scale is powered by batteries. The scale is suitable for use in all establishments, including domestic and those directly connected to the public low-voltage power supply network except for near active HF surgical equipment and the RF shielded room for magnetic resonance imaging.	
Radiated RF emissions CISPR 11	Class B		
Harmonic emissions IEC 61000-3-2	Not applicable		
Voltage fluctuations and flicker emissions IEC 61000-3-3	Not Applicable		
Guidance and manufacture's declaration – electromagnetic immunity			
The scale is intended for use in the electromagnetic environment specified below. The customer or the user of the scale should assure that it is used in such an environment.			
Immunity test	IEC 60601-1-2 test level	Compliance level	Electromagnetic environment-guidance
Electrostatic discharge IEC 61000-4-2	±8kV contact; ±2kV, ±4kV, ±8kV, ±15 kV air	±8kV contact; ±2kV, ±4kV, ±8kV, ±15kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transients/ bursts IEC 61000-4-4	±2kV AC power supply lines; ±1kV DC power/Signal lines. 100 kHz repetition frequency	Not Applicable	Mains power quality should be that of a typical commercial environment.
SURGES IEC 61000-4-5	±0.5kV, ±1kV lines to lines; ±0.5kV, ±1kV, ±2kV lines to earth	Not Applicable	Mains power quality should be that of a typical commercial environment.
Rated power frequency magnetic fields IEC 61000-4-8	30A/m 50HZ OR 60HZ	30A/m 50HZ OR 60HZ	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial environment.
Voltage dips IEC 61000-4-11	0% UT, 0.5 cycle At 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315°; 0% UT, 1 cycle and 70% UT, 25/30 cycle Single phase: at 0°	Not Applicable	Mains power quality should be that of a typical commercial environment. If the user of the scale requires continued operation during power mains interruptions, it is recommended that the scale be powered from an uninterruptible power supply or a battery.
Voltage interruptions IEC 61000-4-11	0% UT, 250/300 cycle	Not Applicable	
NOTE: UT is the a.c. mains voltage prior to application of the test level. E.g.: 25/30 means 25 periods at 50 Hz or 30 periods at 60 Hz. 250/300 means 250 periods at 50 Hz or 300 periods at 60 Hz.			

Guidance and manufacture's declaration – electromagnetic immunity

the scale is intended for use in the electromagnetic environment specified below. The customer or the user of the scale should assure that it is used in such an environment.

Immunity test	IEC 60601-1-2 test level	Compliance level	Electromagnetic environment-guidance
Conducted disturbances induced by RF fields IEC 61000-4-6	3Vrms in 0.15MHz – 80MHz; 6Vrms in ISM and amateur radio bands between 0.15MHz and 80MHz 80% AM at 1kHz	Not Applicable	portable and mobile RF communications equipment should be used no closer to any part of these devices, including cables. Interference may occur in the vicinity of equipment marked with the following symbol: 
Radiated RF EM fields IEC 61000-4-3	10V/m 80MHz – 2.7GHz 80% AM at 1kHz	10V/m 80MHz – 2.7GHz 80% AM at 1kHz	
Proximity magnetic fields IEC 61000-4-39	30 kHz, 8A/m 134,2 kHz, 65A/m 13,56 MHz, 7.5 A/m	30 kHz: 8A/m 134,2 kHz:65A/m 13,56 MHz: 7.5 A/m	

Note 1: At 80 MHz and 800 MHz, the higher frequency range applies.

Note 2: These guidelines may not apply in all situations. Electromagnetic is affected by absorption and reflection from structures, objects and people.

a. The ISM (industrial, scientific and medical) bands between 150 kHz and 80 MHz are 6,765 MHz to 6,795 MHz; 13,553 MHz to 13,567 MHz; 26,957 MHz to 27,283 MHz; and 40,66 MHz to 40,70 MHz

The amateur radio bands between 0,15 MHz and 80 MHz are 1,8 MHz to 2,0 MHz, 3,5 MHz to 4,0 MHz, 5,3 MHz to 5,4 MHz, 7 MHz to 7,3 MHz, 10,1 MHz to 10,15 MHz, 14 MHz to 14,2 MHz, 18,07 MHz to 18,17 MHz, 21,0 MHz to 21,4 MHz, 24,89 MHz to 24,99

MHz, 28,0 MHz to 29,7 MHz and 50,0 MHz to 54,0 MHz

b. Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the These devices is used exceeds the applicable RF compliance level above, the These devices should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the These devices.

c. Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3V/m.

d. This test is applicable only to ME EQUIPMENT and ME SYSTEMS intended for use in the HOME HEALTHCARE ENVIRONMENT.

e. The carrier shall be modulated using a 50 % duty cycle square wave signal.

f. r.m.s., before modulation is applied.

Guidance and manufacture's declaration – electromagnetic immunity

the scale is intended for use in the electromagnetic environment specified below. The customer or the user of the scale should assure that it is used in such an environment.

Immunity test	IEC 60601-1-2 test level	Compliance level	Electromagnetic environment-guidance
Proximity fields from RF wireless communications equipment	See the following table	Complies	/

TEST FREQUENCY (MHZ)	BAND A) (MHZ)	SERVICE A)	MODULATION	IMMUNITY TEST LEVEL (V/M)
385	380 – 390	TETRA 400	PULSE MODULATION B) 18 HZ	27
450	430 – 470	GMRS 460, FRS 460	FM C) ± 5 KHZ DEVIATION 1 KHZ SINE	28
710	704 – 787	LTE BAND 13, 17	PULSE MODULATION B) 217 HZ	9
745				
780				
810				
870	800 – 960	GSM 800/900, TETRA 800, IDEN 820, CDMA 850, LTE BAND 5	PULSE MODULATION B) 18 HZ	28
930				
1720				
1845	1700 – 1990	GSM 1800; CDMA 1900; GSM 1900; DECT; LTE BAND 1, 3, 4, 25; UMTS	PULSE MODULATION B) 217 HZ	28
1970				
2450	2400 – 2570	BLUETOOTH, WLAN, 802.11 B/G/N, RFID 2450, LTE BAND 7	PULSE MODULATION B) 217 HZ	28
5240	5100 – 5800	WLAN 802.11 A/N	PULSE MODULATION B) 217 HZ	9
5500				
5785				

NOTE If necessary to achieve the IMMUNITY TEST LEVEL, the distance between the transmitting antenna and the ME EQUIPMENT or ME SYSTEM may be reduced to 1 m. The 1 m test distance is permitted by IEC 61000-4-3.

- a) For some services, only the uplink frequencies are included.
b) The carrier shall be modulated using a 50 % duty cycle square wave signal.
c) As an alternative to FM modulation, the carrier may be pulse modulated using a 50 % duty cycle square wave signal at 18 Hz. while it does not represent actual modulation, it would be worst case.

5-YEAR LIMITED WARRANTY

Homedics sells its products with the intent that they are free of defects in manufacture and workmanship for a period of 5 years from the date of original purchase, except as noted below. Homedics warrants that its products will be free of defects in material and workmanship under normal use and service. This warranty extends only to consumers and does not extend to Retailers.

To obtain warranty service on your Homedics product, contact a Consumer Relations representative for assistance. Please make sure to have the model number of the product available.

Homedics does not authorize anyone, including, but not limited to, Retailers, the subsequent consumer purchaser of the product from a Retailer or remote purchasers, to obligate Homedics in any way beyond the terms set forth herein. This warranty does not cover damage caused by misuse or abuse; accident; the attachment of any unauthorized accessory; alteration to the product; improper installation; unauthorized repairs or modifications; improper use of electrical/power supply; loss of power; dropped product; malfunction or damage of an operating part from failure to provide manufacturer's recommended maintenance; transportation damage; theft; neglect; vandalism; or environmental conditions; loss of use during the period the product is at a repair facility or otherwise awaiting parts or repair; or any other conditions whatsoever that are beyond the control of Homedics.

This warranty is effective only if the product is purchased and operated in the country in which the product is purchased. A product that requires modifications or adoption to enable it to operate in any other country than the country for which it was designed, manufactured, approved and/or authorized, or repair of products damaged by these modifications is not covered under this warranty.

THE WARRANTY PROVIDED HEREIN SHALL BE THE SOLE AND EXCLUSIVE WARRANTY. THERE SHALL BE NO OTHER WARRANTIES EXPRESS OR IMPLIED INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS OR ANY OTHER OBLIGATION ON THE PART OF THE COMPANY WITH RESPECT TO PRODUCTS COVERED BY THIS WARRANTY. HOMEDICS SHALL HAVE NO LIABILITY FOR ANY INCIDENTAL, CONSEQUENTIAL, OR SPECIAL DAMAGES. IN NO EVENT SHALL THIS WARRANTY REQUIRE MORE THAN THE REPAIR OR REPLACEMENT OF ANY PART OR PARTS WHICH ARE FOUND TO BE DEFECTIVE WITHIN THE EFFECTIVE PERIOD OF THE WARRANTY. NO REFUNDS WILL BE GIVEN. IF REPLACEMENT PARTS FOR DEFECTIVE MATERIALS ARE NOT AVAILABLE, HOMEDICS RESERVES THE RIGHT TO MAKE PRODUCT SUBSTITUTIONS IN LIEU OF REPAIR OR REPLACEMENT.

This warranty does not extend to the purchase of opened, used, repaired, repackaged and/or resealed products, including but not limited to sale of such products on internet auction sites and/or sales of such products by surplus or bulk resellers. Any and all warranties or guarantees shall immediately cease and terminate as to any products or parts thereof which are repaired, replaced, altered, or modified, without the prior express and written consent of Homedics.

This warranty provides you with specific legal rights. You may have additional rights which may vary from state to state and country to country. Because of individual state and country regulations, some of the above limitations and exclusions may not apply to you.

For more information regarding our product line in the USA, please visit www.homedics.com. For Canada, please visit www.homedics.ca.

FOR SERVICE IN THE USA

cservice@homedics.com

8:30am-7:00pm EST

Monday-Friday

1-800-466-3342

FOR SERVICE IN CANADA

cservice@homedics.ca

8:30am-5:00pm EST

Monday-Friday

1-888-225-7378

©2025 Homedics, LLC. All rights reserved. Homedics is a registered trademark of Homedics, LLC. The Bluetooth word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by Homedics is under license. The App Store is a service mark of Apple, Inc. Android, Google Play, and the Google Play logo are trademarks of Google LLC.

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