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3 year limited warranty

# IMPORTANT PRODUCT NOTICES AND SAFETY INSTRUCTIONS

When using your ear thermometer, basic precautions should always be followed. Please read and follow all instructions and warnings before using this product. Save these instructions for future reference.

Please note that this product is intended for household use only and should not serve as a substitute for the advice of a physician or medical professional. Do not use this device for diagnosis or treatment of any health problem or disease. Measurement results are for reference only. Contact your physician if you have or suspect any medical problem.

This ear thermometer is designed for taking temperatures from the ear canal only. Do not use it to take rectal, oral or underarm temperatures. This device is designed for use in human ears only.

- Use ear thermometer only for its intended use.
- This ear thermometer should only be used under the supervision of an adult.
- Prolonged high fever requires medical attention - especially for young children.
- Stand or sit still while taking a temperature.
- Thermometer should be used in a dry ear canal.

- Gentle force should be used when inserting the probe into the ear. Excessive force may injure the ear canal or eardrum.
- Do not expose the monitor to extreme high or low temperatures, very high humidity, or direct sunlight.
- Handle product with care and avoid dropping.
- Do not immerse the unit in water or other liquids.
- Do not disassemble or attempt to repair the unit.
- When not in use, keep thermometer in its storage case and out of the reach of children.
- Never attempt to insert any object into the probe tip.
- Please observe the following regarding batteries:
  - Batteries can be harmful to the thermometer if leakage occurs. When the ear thermometer will not be used for an extended period of time (three months or more), batteries should be removed.
  - A worn battery should be replaced immediately with a new one.
  - Observe caution when inserting battery to avoid insertion with polarities in wrong direction.

Additional product notices and safety instructions are found in the following sections of this manual:

- Important notes regarding temperature measurement (pg. 15)
- Special situations in which the ear thermometer should not be used (pg. 16)
- Care, maintenance, and cleaning (pg. 17)

## Deluxe One-Second Ear Thermometer

### **How does this product work?**

This ear thermometer is a delicate and high-tech electronic device designed to measure body temperature. This unit provides accurate and reliable temperature readings by detecting infrared heat from the eardrum and surrounding tissue.

The thermometer takes 512 scans of the eardrum and surrounding tissue in just one second and displays the calculated temperature. The temperature displayed represents the measured ear canal temperature converted to the more familiar oral equivalent. This is performed using a mathematical adjustment of the ear canal temperature. Note that this reading will not necessarily be the same as a temperature measured orally at the same time.

When consulting a physician, it is important to communicate that the temperature reading was taken in the ear and converted to an oral equivalent.

### **What is normal body temperature? What is a fever?**

Normal body temperature is a range that varies from person to person, by age, and throughout the day. The average normal body temperature is 98.6°F (37°C). Fever is defined as a body temperature elevated over an individual's normal body temperature.

It is important to determine each person's normal body temperature in order to properly assess whether or not a fever is actually present. This range can easily be determined by tracking temperatures on a consistent basis, using the same ear and measuring at the same time each day.

This ear thermometer converts the ear temperature to an oral equivalent and meets industry standards regarding temperature accuracy\*. For temperatures between 96.8°–102.2°F (36.0°–39.0°C), it is accurate to +/- 0.4°F (+/- 0.2°C). See "Product Specifications" (p.19) for more information.

\* ASTM Standard E 1965-98.

## What is the Fever Alarm?

The Fever Alarm is a unique feature that will alert the user anytime a temperature reading is 100°F (37.8°C) or greater with a visual notice on the display (a frowning face) and a double beep audio alert (visual alert only in memory mode). For readings at or above this temperature it is LIKELY that a fever is present.

**Note: A temperature reading below 100°F (37.8°C) does not mean that a fever is not present. Likewise, a temperature greater than or equal to 100°F (37.8°C) does not ensure the presence of a fever.**


Each individual's physiology is different and you should monitor your own body temperature to determine your "normal" temperature range in order to determine when a fever is present. Classification of a temperature as a fever also varies by age.

This product is not intended as a substitute for consultation with a physician. Contact your physician if you feel an illness is present with or without a fever.

A physician should be consulted if any of the following conditions are present: vomiting, diarrhea, changes in breathing, activity, or appetite, irritability, lethargy, or unusual sleepiness.


## DEVICE DESCRIPTION

Figure 1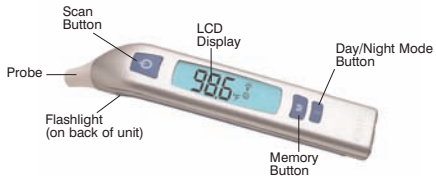

## DESCRIPTION OF DISPLAY SYMBOLS

Figure 2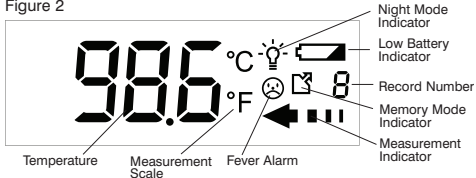

## INSERTING AND REPLACING BATTERIES

1. Remove clear sticker protecting display.
2. Carefully remove the battery cradle as shown in Figure 3.
3. Carefully remove the battery by sliding it out of the cradle as shown in Figure 4. Do not use metal tweezers or a screwdriver to remove the battery.
4. Insert one CR-2032 (3.0 V dc) battery with the plus symbol (+) up.
5. Replace the battery cradle.

Figure 3


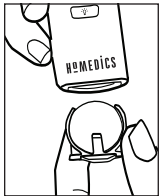
Figure 4

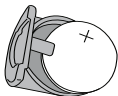Note that the thermometer will display a default temperature of 00 after battery power is restored and until a measurement is taken.

When the low battery symbol appears on the display, the battery should be replaced. When the low battery symbol appears with the “Lo” symbol (Figure 5), the battery must be replaced in order for the ear thermometer to function.

Figure 5

**NOTE:** Do not discard used lithium batteries in trash. Please turn in at an official collection point. If the monitor will not be used for an extended period of time (three months or more), please remove battery to prevent damage to the thermometer from possible battery leakage.

## CHANGING TEMPERATURE SCALE BETWEEN CELSIUS AND FAHRENHEIT

To switch between Celsius and Fahrenheit readings, press and hold the “Scan” button while the unit is on and immediately press the “MEMORY” button to toggle between scales. Once the desired scale is selected, press the “Scan” button to take a new reading or allow the unit to turn off automatically and then turn it back on to view the memory.

## USING NIGHT MODE & DAY MODE

Night Mode is a unique feature that allows a temperature reading to be taken in a dark setting.

Night Mode functions like Day Mode with the addition of an LED flashlight feature to aid in guiding the probe into the ear canal and a display that will illuminate for 5 seconds after measurement.

Press and release the  to select Night Mode or Day Mode.

Day Mode is recommended for general use in order to preserve battery life. Modes can be changed at any time before or after measurement. Do not change modes during measurement as the accuracy of the reading may be affected.

## MEMORY RECALL

This thermometer will automatically store the last 20 measurements in memory. The most recent reading will automatically display when the unit is turned on. Each new temperature reading will also be entered into the memory as the most recent record.

When in memory mode, an arrow on the right will appear with the memory record number as shown below (Fig. 6).

Figure 6

Press “MEMORY” to cycle through the records from number 1 (the most recent reading) to number 20.

Note that when memory is at capacity, the oldest record will be deleted to make room for a new reading.

Memory will be cleared as a result of battery removal or power failure. A default reading of 00 will be displayed when memory is cleared until a new temperature reading is taken.

## HOW TO TAKE A TEMPERATURE READING

1. Always make sure that the waterproof probe has been cleaned with a cotton swab moistened with rubbing alcohol before each use.
2. Press and release the “SCAN” button. All display symbols will appear for approximately one second (Figure 2).
3. When the ear thermometer is ready, the display will next show the results from the last reading.
4. Press and release the “SCAN” button again **with the ear thermometer outside of the ear**.

In DAY MODE: An arrow symbol will appear on the display (Figure 7).

In NIGHT MODE: In addition to the arrow symbol, the flashlight will turn on for approximately two seconds.

Figure 7

5. Insert the probe tip of the ear thermometer into the ear canal within two seconds (Figure 8). It is important that the probe tip be well positioned within the ear canal. Measurement time for the actual reading is approximately one second.

Figure 8

6. A beep will indicate the end of the measurement. Results of the temperature measurement will be displayed. In NIGHT MODE, the display will illuminate for five seconds, and the thermometer will not beep.

If a temperature reading is greater than or equal to 100° F (37.8° C), the fever alarm will display a frowning face and beep twice to alert the user that it is likely that a fever is present (Figure 9). A FEVER MAY BE PRESENT AT A LOWER TEMPERATURE. For more information on the Fever Alarm feature, please refer to the “What is the Fever Alarm?” section of this manual (page 6).

Figure 9

The measuring range of this ear thermometer is 50° – 122° F (10° – 50° C). If a measurement is below this range, the unit will display “Lo” (Figure 10). If above this range, the unit will display “Hi” (Figure 11).

Figure 10

Figure 11

The unit will display “Err” if the operating temperature is not within the permissible range for measurement (see details on pg. 18) (Figure 12).

Figure 12

7. The unit will automatically turn off after 30 seconds of non-use.

## IMPORTANT NOTES REGARDING TEMPERATURE MEASUREMENT

- The measurement results that users receive are for reference only. If users have any health concerns, please consult a physician.
- It is not necessary to perform an “ear tug” or pull on the ear before taking a temperature measurement.
- For additional measurements, users should wait at least 5 seconds between readings to ensure accuracy.
- Use the ear thermometer at room temperature, as extremes in environmental temperature can impact accuracy. Unit should be stored at room temperature. Before use, the unit must be at room temperature for at least 30 minutes.
- Clean the waterproof probe with a cotton swab moistened with rubbing alcohol between readings. Always keep rubbing alcohol/cleaners away from children.
- It is recommended that you take multiple readings (three or more) in the same ear and use the highest reading in the following special circumstances:
  - When taking the temperature of an infant younger than three months.
  - When taking the temperature of children under the age of three who have special conditions, such as compromised immune systems, and for whom the absence or presence of a fever is important.
  - When first learning how to use thermometer you should take enough readings to get consistent results.

**Note:** Slight temperature variations may be observed (+/- 0.3 – 0.5°F or +/- 0.2 – 0.3°C) in consecutive measurements. This is common to all types of thermometers.

- Always take the temperature in the same ear, as readings may differ from one ear to another.
- The ear must be free from excessive earwax buildup or other obstructions in order to take an accurate reading. Note that you should never attempt to clean inside ears, however, as this may result in damage to the eardrum or surrounding tissues. Remove only excess earwax that can be reached with a washcloth. Consult a physician if you suspect excess earwax is present within the ear.
- Ear temperatures may be influenced by external factors, including:
  - An ear that has been rested upon or covered
  - General exposure to extreme hot or cold temperatures
  - Recent bathing or swimming
  - Use of hearing aids or ear plugs

It is recommended that measurement occur at least 20 minutes after these circumstances are no longer present.

- This ear thermometer is for use in human ears only. Do not use for oral, rectal, or axillary temperature measurement.

### **SPECIAL SITUATIONS IN WHICH THE EAR THERMOMETER SHOULD NOT BE USED**

- Pain of any kind is noted during use.
- Blood or drainage is present in the external ear canal.

- Deformities of the ear canal or face exist that prohibit the thermometer from being properly inserted into the ear canal.
- Ear medication or ear drops have been placed in the ear canal.
- Ear infection of any kind is present.
- Individual has been bathing or swimming, as this may lead to injury of ear tissue, as well as inaccurate readings.

### **CARE, MAINTENANCE & CLEANING**

- Clean the unit carefully with a soft, dry cloth. Do not use any gas, strong detergent, or solvents to clean the device.
- Clean the waterproof probe with a cotton swab moistened with rubbing alcohol.
- Never attempt to insert any object into the probe tip.
- Do not immerse the unit in water as this will result in damage to the unit. Waterproof portion of unit is limited to probe tip.
- Do not subject the ear thermometer to extreme temperatures, humidity, moisture, and direct sunlight.
- Do not disassemble the ear thermometer. If in need of repair, refer to the warranty section of this manual.
- Do not subject the ear thermometer to extreme shocks (do not drop on floor).
- When not in use, keep thermometer in its storage case and out of the reach of children. If the thermometer will not be used for three months or more, remove battery.
- The thermometer is calibrated by the manufacturer. If you have questions about the use of your thermometer, please contact HoMedics Consumer Relations at 1-800-466-3342.

## TROUBLESHOOTING/ERROR CODES

Condition	Possible Cause	Corrective Action
Consistent low temperature readings ("Lo")	Temperature reading is below measurement range (below 50°F/10°C).	Product Design-Measurement range is 50° – 122°F (10° – 50°C). Contact your physician if you have or suspect any medical problems.
	The probe is not positioned properly in ear.	Make sure probe is properly positioned against the opening of the ear canal
	The waterproof probe is dirty.	Clean the probe with a cotton swab moistened with alcohol.
Consistent high temperature readings ("Hi")	Temperature reading is above measurement range (above 122°F/50°C).	Product Design-Measurement range is 50° – 122°F (10° – 50°C). Contact your physician if you have or suspect any medical problems.
Consistent error messages ("Err")	Operating temperature is not within the permissible range for measurement.	Product Design-Operating temperature is 60.8° – 95°F (16° – 35°C).
Low battery warning	Battery power is low.	Replace battery (see "Inserting and Replacing Batteries").
Low battery warning with "Lo" symbol	Battery power is too low for the thermometer to function.	Replace battery immediately (see "Inserting and Replacing Batteries").

## PRODUCT SPECIFICATIONS

Model Number	Model Number TE-120
Power Source	DC 3 Volt (lithium battery CR2032)
Battery Life	1,700 Measurements in day mode or 700 Measurements in night mode.
Sensor	Thermopile
Temperature Indication	In 4 digits, every 0.1°F or °C
Accuracy	+/- 0.4°F/0.2°C between 96.8° – 102.2°F (36.0° – 39.0°C) +/- 0.5°F/0.3°C less than 96.8°F (36.0°C) or greater than 102.2°F (39.0°C)
Measurement Range	50° – 122°F (10° – 50°C)
Weight	0.12 lbs (with battery)
Operating Temperature	60.8° – 95°F (16° – 35°C)
Operating Humidity	Relative humidity up to 95% (non-condensing)
Storage Temperature	-4° – 122°F (-20° – 50°C)
Storage Humidity	Relative humidity up to 95% (non-condensing)
Contents	Ear Thermometer & Battery, Storage Case, Instruction Manual

**NOTE:** These specifications are subject to change without notice. This ear thermometer meets requirements established in ASTM Standard E 1965-98.

**HoMEDICS**  
THERA:P

#### LIMITED THREE YEAR WARRANTY

HoMedics sells its products with the intent that they are free of defects in manufacture and workmanship for a period of one year from the date of original purchase, except as noted below. HoMedics warrants that its products will be free of defects in material and workmanship under normal use and service. This warranty extends only to consumers and does not extend to Retailers.

To obtain warranty service on your HoMedics product, mail the product and your dated sales receipt (as proof of purchase), postpaid, to the following address:

HoMedics Consumer Relations  
Service Center Dept. 168  
3000 Pontiac Trail  
Commerce Township, MI 48390

No COD's will be accepted.

HoMedics does not authorize anyone, including, but not limited to, Retailers, the subsequent consumer purchaser of the product from a Retailer or remote purchasers, to obligate HoMedics in any way beyond the terms set forth herein. This warranty does not cover damage caused by misuse or abuse; accident; the attachment of any unauthorized accessory; alteration to the product; improper installation; unauthorized repairs or modifications; improper use of electrical/power supply; loss of power; dropped product; malfunction or damage of an operating part from failure to provide manufacturer's recommended maintenance; transportation damage; theft; neglect; vandalism; or environmental conditions; loss of use during the period the product is at a repair facility or otherwise awaiting parts or repair; or any other conditions whatsoever that are beyond the control of HoMedics.

This warranty is effective only if the product is purchased and operated in the country in which the product is purchased. A product that requires modifications or adoption to enable it to operate in any other country than the country for which it was designed, manufactured, approved and/or authorized, or repair of products damaged by these modifications is not covered under this warranty.

THE WARRANTY PROVIDED HEREIN SHALL BE THE SOLE AND EXCLUSIVE WARRANTY. THERE SHALL BE NO OTHER WARRANTIES EXPRESS OR IMPLIED INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS OR ANY OTHER OBLIGATION ON THE PART OF THE COMPANY WITH RESPECT TO PRODUCTS COVERED BY THIS WARRANTY. HoMEDICS SHALL HAVE NO LIABILITY FOR ANY INCIDENTAL, CONSEQUENTIAL OR SPECIAL DAMAGES. IN NO EVENT SHALL THIS WARRANTY REQUIRE MORE THAN THE REPAIR OR REPLACEMENT OF ANY PART OR PARTS WHICH ARE FOUND TO BE DEFECTIVE WITHIN THE EFFECTIVE PERIOD OF THE WARRANTY. NO REFUNDS WILL BE GIVEN. IF REPLACEMENT PARTS FOR DEFECTIVE MATERIALS ARE NOT AVAILABLE, HoMEDICS RESERVES THE RIGHT TO MAKE PRODUCT SUBSTITUTIONS IN LIEU OF REPAIR OR REPLACEMENT.

This warranty does not extend to the purchase of opened, used, repaired, repackaged and/or resealed products, including but not limited to sale of such products on Internet auction sites and/or sales of such products by surplus or bulk resellers. Any and all warranties or guarantees shall immediately cease and terminate as to any products or parts thereof which are repaired, replaced, altered, or modified, without the prior express and written consent of HoMedics.

This warranty provides you with specific legal rights. You may have additional rights which may vary from country to country. Because of individual country regulations, some of the above limitations and exclusions may not apply to you.

For more information regarding our product line in the USA, please visit: [www.homedics.com](http://www.homedics.com)

©2008 HoMedics, Inc. and its affiliated companies, all rights reserved. HoMedics® is a registered trademark of HoMedics, Inc. and its affiliated companies. TheraP™ is a trademark of HoMedics, Inc. and its affiliated companies. All rights reserved.

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